REF:97055447 REV.3

# S220TR-S200 QUICK REFERENCE

EN | RU | TR | RO

Making Your Life Better.

**BU Medical Equipment**  
**Sede Legale ed Amministrativa / Headquarters**  
CEFLA s.c.  
Via Selice Provinciale, 23/a  
40026 Imola (BO) - Italy  
t. +39 0542 653111  
f. +39 0542 653344  
sternweber@sternweber.com  
sternweber.com

**Stabilimento / Plant**  
Via Bicocca, 14/C  
40026 Imola (BO) - Italy  
t. +39 0542 653441  
f. +39 0542 653601

STERNWEBER.COM

DENTAL CHAIR • КРЕСЛО • KOLTUK • SCAUN

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**EN** Dental chair position set manually.

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**EN** Independent water supply tank in reserve.

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CAREFULLY READ THE COMPLETE USER MANUAL AVAILABLE IN PDF FORMAT AND DOWNLOADABLE FROM THE DOWNLOAD AREA OF THE WEBSITE [www.sternweber.com](http://www.sternweber.com). IN PARTICULAR, CAREFULLY READ THE SAFETY WARNINGS AND THE CONDITIONS FOR USE OF THE PRODUCT. YOU CAN REQUEST A PAPER COPY OF THE USER MANUAL TO YOUR DEALER OR BY SENDING AN E-MAIL TO THE FOLLOWING ADDRESS: [request@sternweber.com](mailto:request@sternweber.com)

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## INTENDED USE AND MODE OF USE

The dental units described in this quick start guide are medical devices intended for dental treatment. This equipment must be used only by adequately trained personnel (dentists and paramedics).

### Contraindications.

The device is not intended for the following uses and/or applications:

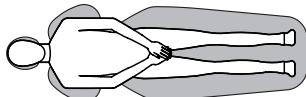
- Use on unintended anatomical areas;
- Use on patients whose weight exceeds the maximum permissible load indicated in the user manual;
- Use on non-alert or uncooperative patients or patients unable to maintain their position for the duration of the operation;
- Use by an unauthorised dental practitioner;
- Use by an operator not trained on the device.

Disclaimer: the trademarks of the endodontic drills are not the property of CEFLA s.c. or any of its affiliated companies.

## SAFETY WARNINGS

- Read the full user manual carefully before using the dental unit.
- This device may not be modified in any way without the authorisation of the manufacturer. CEFLA s.c. shall not be held liable for any personal injury or property damage arising from failure to heed the following clause.
- The device delivered is not sterile.
- The installation of all the devices is always permanent, CEFLA s.c. shall not be held liable for any personal injury or property damage arising from failure to heed the following clause.  
The application of the integrated X-ray device requires a correct fixing of the dental unit to the ground. A correct fixing of the dental unit to the ground is required to enable the extra rotation.
- The equipment must be assembled, repaired, calibrated and upgraded only by technicians authorised by CEFLA s.c.
- In case of an auxiliary electric socket, do not exceed the max. value indicated on the nameplate.
- The equipment is not protected against liquid penetration (IPX0).
- The equipment is not suitable for use in the presence of a mixture of flammable anaesthetic gas with oxygen or nitrous oxide.
- The user must be present at all times when the equipment is turned on or ready for start-up. In particular, never leave the equipment unattended in the presence of children / the mentally disabled or other unauthorised personnel in general. Any accompanying person must stay outside the area in which the treatment is performed and in any case under the responsibility of the operator. The operating area refers to the space around the dental unit plus 1.5 meters.
- Patients with pacemakers and/or hearing aids: when treating patients with pacemakers and/or hearing aids, take into consideration the effects the instruments may have on pacemakers and/or hearing aids. Please refer to the technical scientific literature available on this subject.
- Implants: if the dental unit is used for implants operations using separate equipment designed for this purpose, you are recommended to power off the dental chair to avoid unwanted movements resulting from faults and/or accidental start up of the movement controls.
- Maximum permissible loads, **do not exceed these values:**  
Maximum permissible load on the dental chair: see user manual.  
Dental chair armrest: 68 Kg.  
Dental chair headrest: 30 Kg.  
Tray-holder attached to the dentist's board: 2Kg distributed.  
Tray-holder attached to the assistant's board: 1 kg distributed.  
Auxiliary tray-holder: 3.5 Kg without negatoscope or 2.5 Kg with negatoscope.
- Replacing the drills:  
Operate the turbine release and contra angle devices only once the drill has come to a complete stop. Failure to do so will result in damaging the locking system and drills could be released and cause injury. Exclusively use high-quality drills with a connection rod having a calibrated diameter.  
Drills and various instruments attached to the handpieces must comply with Biocompatibility Standard ISO 10993.
- Dental chair movement: make sure that the patient is ready to collaborate. Ask him/her to keep his/her hands and feet close, avoiding incorrect postures. Pay attention to the patient and the staff during dental chair movement.

- Headrest: do not attempt to move headrest while patient is resting against it.
- For operators in Europe: report any serious accident occurring in relation to the device to CEFLA s.c. and to the competent authority of the Member State in which the user and/or patient is located.

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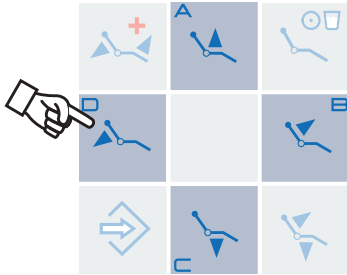
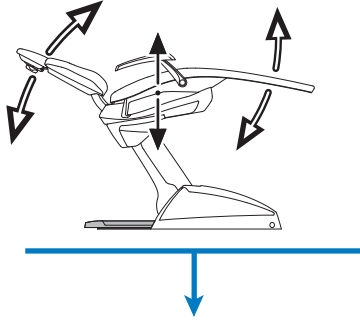
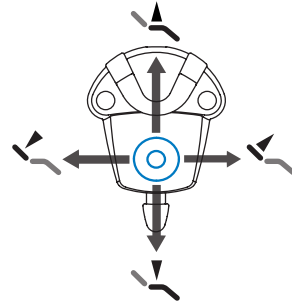
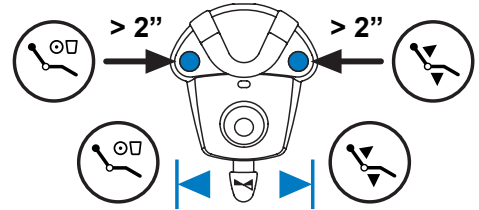
**WARNING! • ВНИМАНИЕ ! • DİKKAT! • ATENÇJE!**

Use the devices below when movement of the equipment needs to be blocked:

- Dental chair buttons: by pressing any of the dental chair buttons all movements will be stopped.
- Foot control: when the foot control is actuated, all movements of the equipment are blocked.
- Dental chair foot board (dental chair 300 P - 320 P TR) or safety pedal (dental chair 280 P - 380 P): when the dental chair foot board or safety pedal are activated, all movements of the equipment are blocked.

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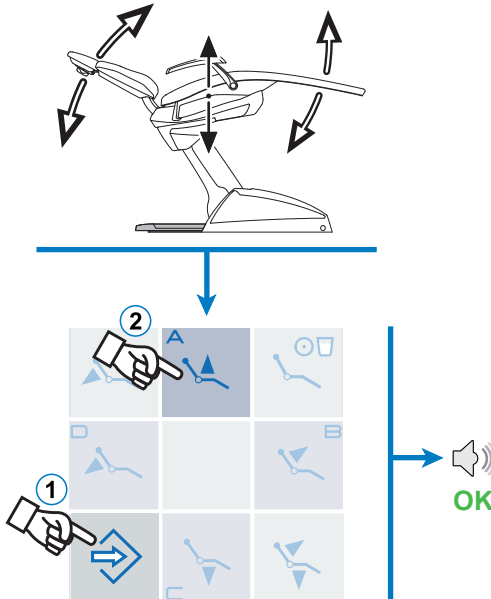
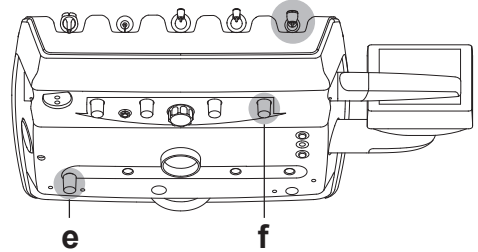
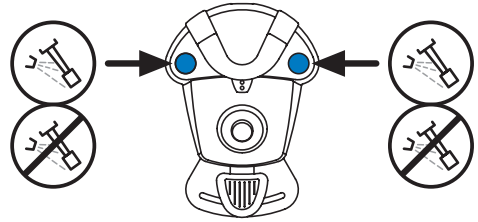
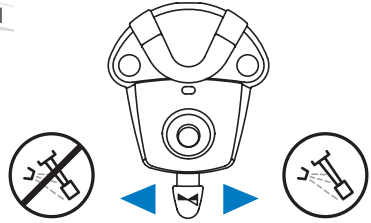
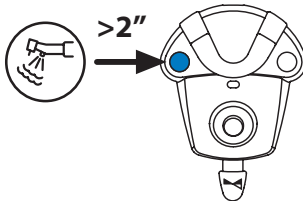
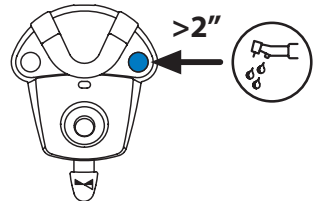
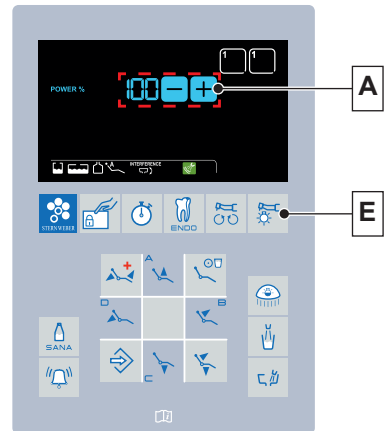
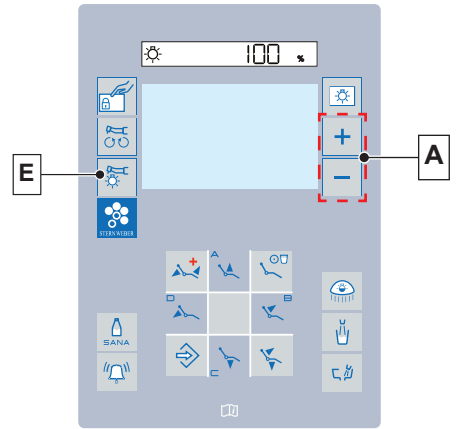
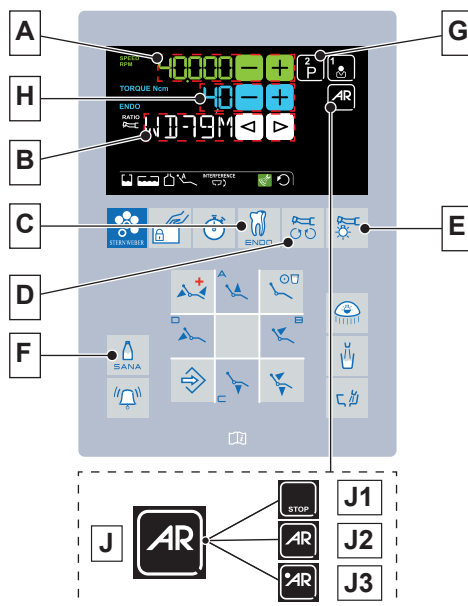
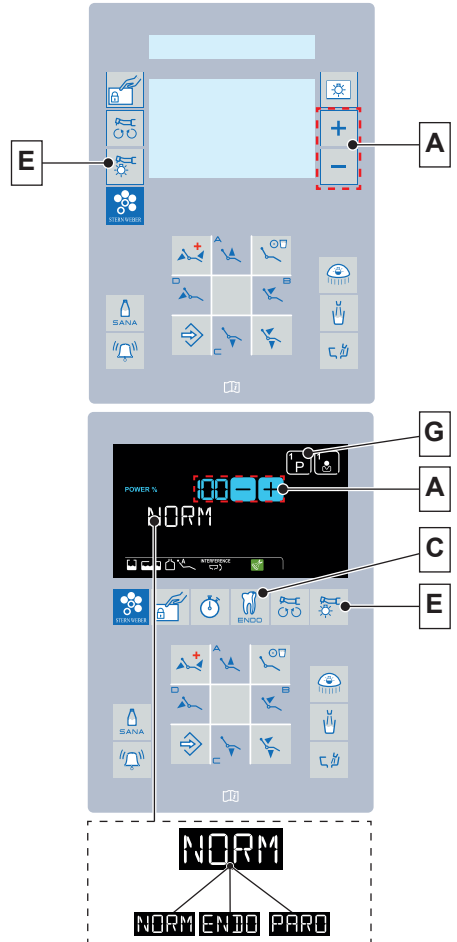
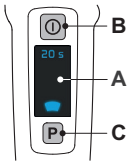
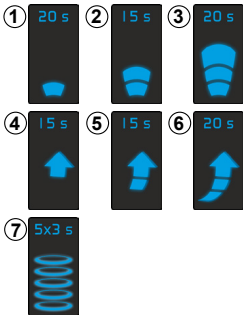
**WARNING! • ВНИМАНИЕ ! • DİKKATİ • ATENȚIE!**

• The LED is a Class 2 light source in accordance with IEC 62471 standard. DO NOT STARE AT THE BEAM. The light emitted may cause eye injury in case of direct radiation without eye protection. The light emitted may damage soft tissues (oral cavity mucous membrane, gums, skin). Be extremely careful to aim the light precisely on the material to be cured. Operating mode: work 2 consecutive cycles, rest 60 seconds.

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**WARNING! • ВНИМАНИЕ ! • DİKKATİ • ATENȚIE!**

Please read carefully the safety warnings and the conditions of use of the product.

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**In FREEZE mode**: it allows scrolling through pages with frozen images.

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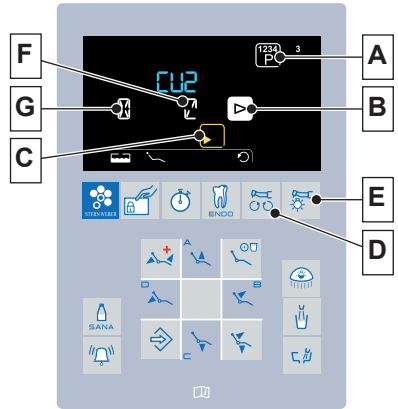
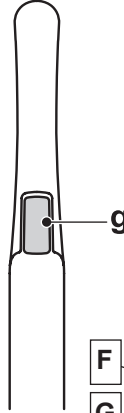
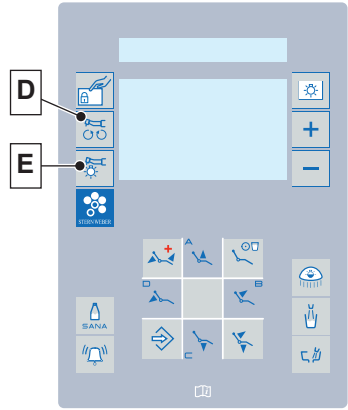
**⚠ WARNING! • ВНИМАНИЕ! • DİKKAT! • ATENȚIE!**

Do not aim the intermittent light beam at the operator's or patient's eyes during operation.

During continuous use (example, more than 10 consecutive minutes), the temperature of the camera's tip usually increases significantly; if this is uncomfortable, put the handpiece in its holder for a few minutes to allow the light source to cool down. When the camera needs to be used for a prolonged time, reduce light brightness using the cursor on the OSD Control Panel.

If left running for extended periods, make sure the temperature of the tip is acceptable before attempting to use the camera by briefly touching the clear plastic part with your fingertip, being careful not to touch the lens in the middle.

Disposable infection control sheaths must be used with the device and changed for each new patient.

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Su düğmesi.

Buton apă.

**f** 6-function syringe only: heat/cold selector.

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6-function syringe only: Heat/cold indicator LED.

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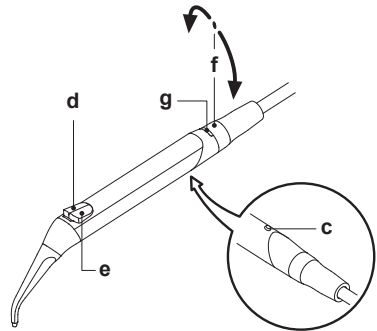
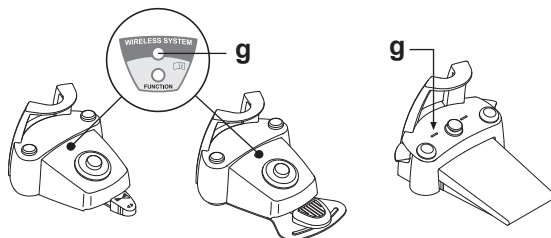
LED COLOUR	DESCRIPTION (CABLE DISCONNECTED)	DESCRIPTION (CABLE CONNECTED)
<b>GREEN</b>	Battery charge (>75%)	Battery charged
<b>ORANGE</b>	Battery charge (<50%)	Battery charging
<b>RED</b>	Battery needs charging (<25%)	Battery charge error
<b>Off</b>	Battery flat	Dental unit off or foot control fault

LED FLASHING	DESCRIPTION
<b>Slow</b>	Connection active in wireless mode
<b>Fast</b>	Connection active with charging cable inserted
<b>Double</b>	Connection search
<b>On fixed</b>	Communication error


## RECHARGING THE BATTERY • ЗАРЯДКА АККУМУЛЯТОРА • AKÜNÜN ŞARJ EDİLMESİ • ÎNCĂRCAREA BATERIEI

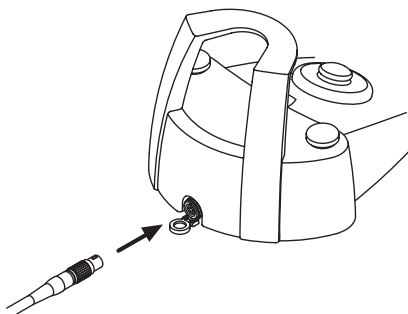When necessary, recharge the Wireless foot control batteries by connecting the cable to the base of the dental unit. Connect it with the dental unit off.

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**WARNING! • ВНИМАНИЕ! • DİKKAT! • ATENȚIE!**

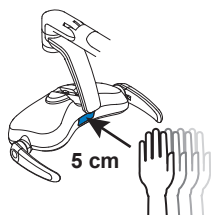
The instructions for use and maintenance of the lamps are available in PDF format and can be downloaded from the download area of the [www.sternweber.com](http://www.sternweber.com).

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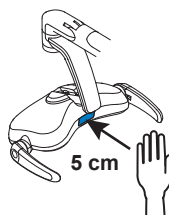
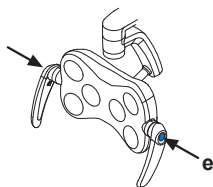
**e** Button • Кнопка • Düğme • Buton.

**Brief pressure:** STANDARD / COMPOSITE operation.

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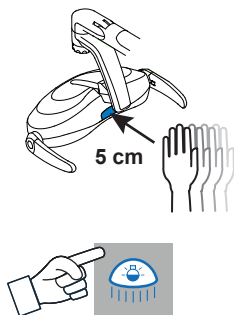
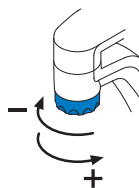
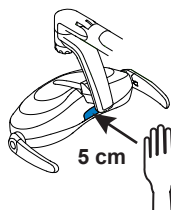
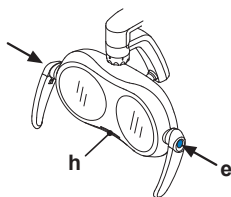
**Long press:** change COLOR TEMPERATURE.

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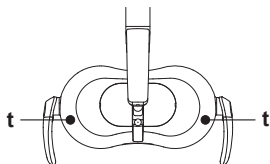
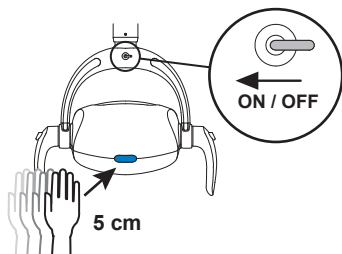
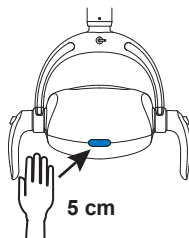
WARNING	SOLUTION
W001	End movements, lay down instruments
W050	To start a disinfection cycle, all disinfection instruments must be in a rest position.
W051	Select the instrument, cup or suction tubes, then activate the disinfection cycle.
W057	To restart the disinfection cycle after a lock-out, use the same instruments extracted during the locked cycle.
W059	Select the instrument or cup, then start the disinfection cycle.
W060	Fill the H <sub>2</sub> O <sub>2</sub> tank.
W061	None.
W062	None.
W063	Fill the H <sub>2</sub> O tank.
W064	Fill the H <sub>2</sub> O <sub>2</sub> tank.
W065	Check that dentist's board water cocks are open, if the problem persists, contact the technical service department.
W066	Select mains water supply.
W067 W068	Open main water supply, if the problem persists, contact the Technical Service department.
W100 W700	Release the pedal control then activate the instrument.
W101 W701	Put the instruments back in place.
W102 W702	Check connections. If the problem persists, contact the technical service department.
W103 W703	Repeat the Flushing cycle request with the instruments extracted.
W104 W704	Fill the H <sub>2</sub> O tank and repeat the Flushing request. If the problem persists, contact technical service department.
W105 W705	Repeat the Flushing cycle.
W106 W706	Repeat the Flushing cycle request with mains water selected.
W150	Put suction tubes back in place.
W151	Put instruments back in place.
W200	Remove the obstacle. If the problem persists, contact the technical service department.
W206	Deselect the supply from bottle and repeat the suction tube washing request with mains water selected.
W208	None.
W209	Check that suction tube terminal is open and filters are clean. If the problem persists, contact the technical service department.
W210	None.
W211	End Flushing or BIOSTER cycle.
W253	Contact technical service department.
W352	Connect the power cable to the foot control.

WARNING	SOLUTION
W353	Disconnect the power cable from the foot control.
W355	Activate the wireless foot control for at least 1 second.
W400	Remove the obstacle. If the problem persists, contact the technical service department.
W401	Remove the obstacle. If the problem persists, contact the technical service department.
W403	Wait for the end of the disinfection cycle, then put the instruments back in place to make movements available again.
W406	Manually move the bowl out of the interference area.
W409	Remove the obstacle. If the problem persists, contact the technical service department.
W410	Remove the obstacle. If the problem persists, contact the technical service department.
W411	Put instrument back in the rest position to make automatic movement programmes available again.
W412	To move the dental chair manually, disable the instrument. To activate automatic movements, disable the instrument and put it back in place.
W413	Remove the dental chair lock using the suitable button.
W415	Remove the obstacle. If the problem persists, contact the technical service department.
W416	Remove the obstacle. If the problem persists, contact the technical service department.
W417	Remove the obstacle. If the problem persists, contact the technical service department.
W424	Lower the dental chair so that it is out of the interference position, then request the bowl movement.
W428	Remove the obstacle. If the problem persists, contact the technical service department.
W430	Wait for the motor to cool down.
W431	Restart.

Error messages (EXXX) indicate situations that cannot be fixed by the user and require the intervention of the Technical Service.

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**WARNING!**

It indicates a condition in which the failure to observe the instructions may result in equipment damage or injury to the user and/or patient.

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Biological hazard.

It provides information about possible risks of contamination deriving from contact with fluids, storage of infected biological waste.

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Pericol de strivire a piciorului.

Equipment equivalent to Class 2 light source.

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Pericol de răsturnare.

*If a different pedal or a pedal with wireless technology is used, refer to the user manual*

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