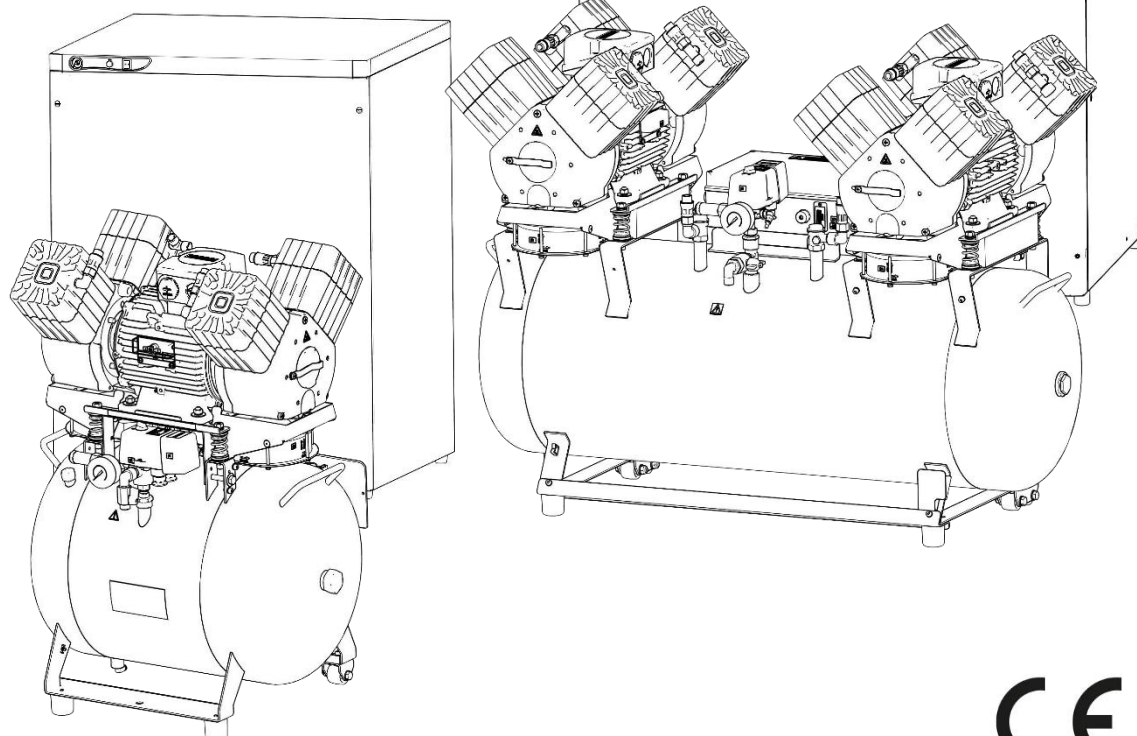
DK50 4VR/50

DK50 2X4VR/110

- EN USER MANUAL
- DE BENUTZERHANDBUCH
- FR MANUEL DE L'UTILISATEUR
- RU РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ
- PL INSTRUKCJA OBSŁUGI
- SK NÁVOD NA POUŽITIE
- CS NÁVOD K POUŽITÍ

COMPRESSOR
KOMPRESSOR
COMPRESSEUR
КОМПРЕССОР
SPREŽARKA
KOMPRESOR

DK50 4VR/50
DK50 2x4VR/110

EKOM spol. s r. o.
Priemyselná 5031/18
SK-921 01 Piešťany
Slovak Republic
tel.: +421 33 7967255
fax: +421 33 7967223

www.ekom.sk
email: ekom@ekom.sk

DATE OF LAST REVISION
DATUM DER LETZTEN ÜBERARBEITUNG
DATE DE LA DERNIÈRE RÉVISION
ДАТА ПОСЛЕДНЕЙ РЕДАКЦИИ
DATA OSTATNIEJ AKTUALIZACJI
DÁTUM POSLEDNEJ REVÍZIE
DATUM POSLEDNÍ REVIZE

05/2022

NP-DK50 4VR 50, DK50
2x4VR 110-MD-9_05-2022
112000445-000

EN	<u>CONTENTS..... 5</u>
DE	<u>INHALT 58</u>
FR	<u>TABLES DES MATIÈRES 112</u>
RU	<u>ОГЛАВЛЕНИЕ 165</u>
PL	<u>SPIS TREŚCI 220</u>
SK	<u>OBSAH 275</u>
CS	<u>OBSAH 328</u>

CONTENTS

IMPORTANT INFORMATION	6
1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES	6
2. INTENDED USE	6
3. CONTRAINDICATIONS AND SIDE-EFFECTS	6
4. SYMBOLS.....	6
5. WARNINGS	7
6. STORAGE AND TRANSPORT CINDITIONS.....	9
PRODUCT DESCRIPTION	10
7. VARIANTS	10
8. ACCESSORIES	11
9. PRODUCT FUNCTION	12
TECHNICAL DATA	18
INSTALLATION	25
10. INSTALLATION CONDITIONS	25
11. COMPRESSOR ASSEMBLY	25
12. PNEUMATIC CONNECTION	30
13. ELECTRICAL CONNECTION	32
14. COMMISSIONING	33
15. PNEUMATIC AND ELECTRICAL DIAGRAMS	34
OPERATION	44
16. SWITCHING ON THE COMPRESSOR.....	45
17. SWITCHING OFF THE COMPRESSOR.....	45
PRODUCT MAINTENANCE	46
18. PRODUCT MAINTENANCE.....	46
19. LONG-TERM SHUTDOWN.....	55
20. DISPOSAL OF DEVICE	55
TROUBLESHOOTING	56
21. REPAIR SERVICE	57
ANNEX	381
22. INSTALLATION RECORD	381

IMPORTANT INFORMATION

1. CONFORMITY WITH THE REQUIREMENTS OF EUROPEAN UNION DIRECTIVES

This product conforms to the requirements of the Regulation (EU) on medical devices (MDR 2017/745) and is safe for the intended use if all safety instructions are followed.

2. INTENDED USE

The compressor is used as a source of clean, oil-free compressed air to power active medical devices where the parameters and properties of the compressed air are suitable for the specific application.a

Compressed air supplied by the compressor is unsuitable for use with artificial lung ventilation devices without further filtration

Any other use of the product beyond its intended use is considered an incorrect use. The manufacturer is not liable for any damages or injuries resulting from the incorrect use.

3. CONTRAINDICATIONS AND SIDE-EFFECTS

There are no contraindications or side-effects known.

4. SYMBOLS

The following symbols and marks are used in the User manual, on the device and its packaging:

General warning

Warning - risk of electric shock

Warning - compressor is controlled automatically

Warning - hot surface

General caution

Read the operating instructions

Refer to instruction manual

CE – marking

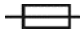
Medical device

Serial number

Protecting earthing

	Fuse
	Package handling label – fragile
	Package handling label – this side up
	Package handling label – keep dry
	Package handling label – temperature limits
	Package handling label – limited stacking
	Package label – recyclable material
	Manufacturer

5. WARNINGS

The product is designed and manufactured to be safe for the user and the surrounding environment when used in the defined manner. Keep the following warnings in mind.

5.1. General warnings

READ THE USER MANUAL CAREFULLY BEFORE USING THE DEVICE AND KEEP IT FOR FUTURE USE!

- The user manual supports the correct installation, operation and maintenance of the product. Careful review of this manual will provide the information necessary for the correct operation of the product for its intended use.
- Original packaging should be retained for the return of the device. Only the original packaging ensures protection of the device during transport. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.
- This warranty does not cover damages originating from the use of accessories other than those specified or recommended by the manufacturer.
- The manufacturer only guarantees the safety, reliability and function of the device if:
 - installation, new settings, changes, expansion, and repairs are performed by the manufacturer or an organisation authorised by the manufacturer
 - the product is used pursuant to the user manual
- The user manual corresponds to the configuration of the product and its compliance with the applicable safety and technical standards at the time of printing. The manufacturer reserves all rights for the protection of its configuration, methods and names.
- Translation of the user manual is performed in accordance with the best available knowledge. The Slovak version is to be used in the event of any uncertainties.
- This user manual is the original instructions. Translation is performed in accordance with the best available knowledge.

5.2. General safety warnings

The manufacturer designed and manufactured the product to minimise all risks when used correctly for the intended use. The manufacturer considers it its obligation to lay down the following general safety precautions.

- Use and operation of the product must comply with all laws and local regulations valid in the place of use. The operator and user are responsible for following all the appropriate regulations in the interests of performing work safely.
- Only the use of original parts guarantees the safety of operating personnel and the flawless operation of the product itself. Only the accessories and parts mentioned in the technical documentation or expressly approved by the manufacturer should be used.
- The operator must ensure that the device is functioning correctly and safely before every use.
- The user must be familiar with the operation of the device.
- Do not use the product in environments with a risk of explosion.
- The user must inform the supplier immediately if any problem directly related to the operation of the device occurs.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

5.3. Safety warnings on protection from electric current

- The device must only be connected to a properly installed, earthed socket.
- Before the product is plugged in, ensure that the mains voltage and frequency stated on the product are in compliance with the values of the mains.
- Prior to putting the device into operation check for any damage to the connected pneumatic lines and electrical wiring. Replace damaged pneumatic lines and electrical wirings immediately.
- Immediately disconnect the product from the mains (remove power cord from the socket) in hazardous situations or in the case of a technical malfunction.
- During all repairs and maintenance, ensure that:
 - the mains plug is removed from the power socket
 - pressure is vented from the air tank and pipes
- The product shall only be installed by a qualified technician.

6. STORAGE AND TRANSPORT CINDITIONS

The compressor is shipped from the manufacturer in transport packaging. This protects the product from damage during transport.

Potential for damage to pneumatic components.

The compressor must be transported only when all air has been vented. Before moving or transporting the compressor, release all the air pressure from the tank and pressure hoses, from dryer chambers and drain condensate from the tank and from the condensate separator on the dryer.

Keep the original factory packaging in case the device needs to be returned Use the original factory packaging during transport as it provides optimum protection for the product. If it is necessary to return the product during the warranty period, the manufacturer is not liable for damages caused by improper packaging.

The compressor is shipped in a vertical position and must be secured using transport straps.

Protect the compressor from humid and dirty environments and extreme temperatures during transport and storage. Do not store near any volatile chemical substances.

If not, please dispose of the original packaging material in an environmentally-friendly way. The packaging cardboard can be recycled with old paper.

Storing or shipping the equipment in any conditions other than those specified below is prohibited.

6.1. Ambient conditions

Products may only be stored and transported in vehicles that are free of any traces of volatile chemicals under the following climactic conditions:

Temperature

–25°C to +55°C

Relative humidity


max. 90% (non-condensing)

PRODUCT DESCRIPTION


7. VARIANTS

The compressor is manufactured according to its intended application in the following variants:


DK50 4VR/50	Compressor for installation in areas where operations will not disturb the surroundings
DK50 4VR/50/M	Compressor with a membrane air dryer
DK50 4VR/50S	Compressor in a cabinet with efficient soundproofing
DK50 4VR/50S/M	Compressor with membrane dryer in a cabinet
Cabinet S50R	Used to reduce the level of noise generated by the compressor
DK50 2x4VR/110	Compressor for installation in areas where operations will not disturb the surroundings
DK50 2x4VR/110/M	Compressor with a membrane air dryer
DK50 2x4VR/110S	Compressor in a cabinet with efficient soundproofing
DK50 2x4VR/110S/M	Compressor with membrane dryer in a cabinet
Cabinet S110R	Used to reduce the level of noise generated by the compressor

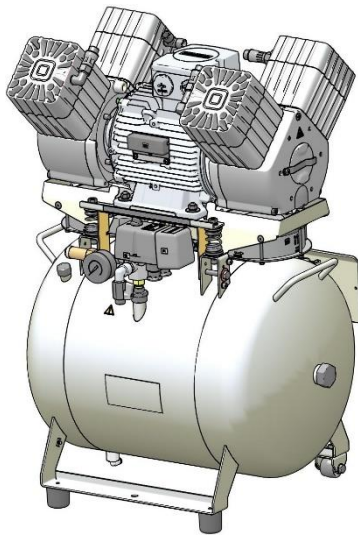
DK50 4VR/50

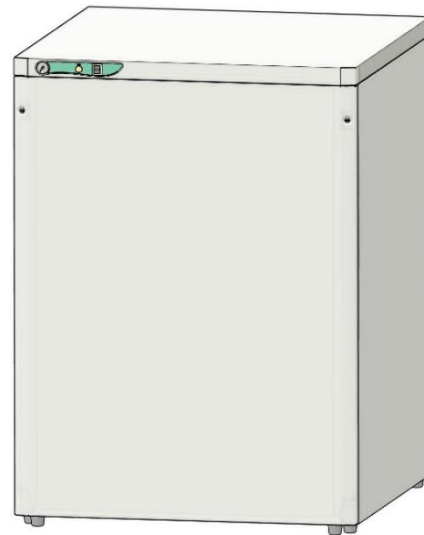S50R
DK50 4VR/50S

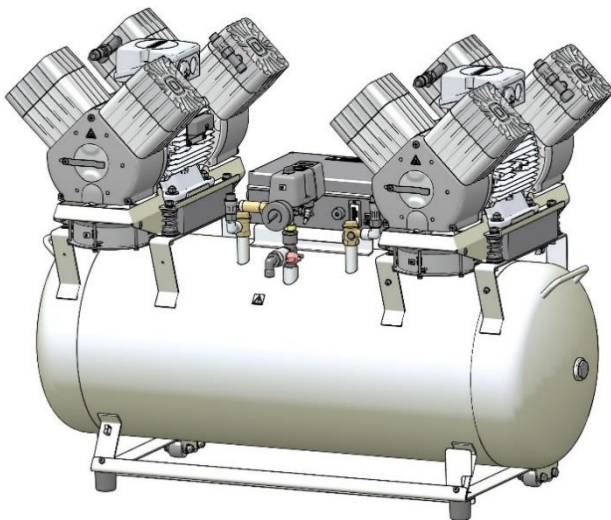DK50 2x4VR/110

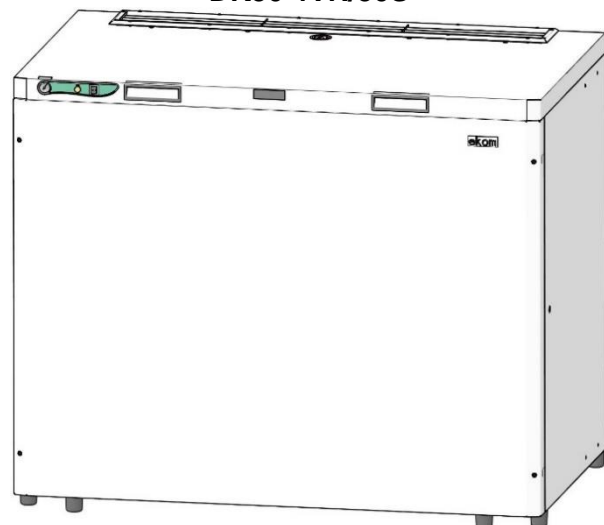S110R
DK50 2x4VR/110S

8. ACCESSORIES

Accessories that are not included in the standard order must be ordered separately.

Automatic condensate drain

The automatic condensate drain (AOK) automatically drains condensate from the compressor's air tank based on a pre-set time interval. The condensate drain (AOK) is a suitable accessory for compressor models without dryers.

Type	Use	Article number
AOK 18	DK50 4VR/50	604014082-000
AOK 19	DK50 2x4VR/110	604014083-000

Set of compressed air outlet filters

The compressor may be equipped with a set of filters if specified. The filter set may be equipped with an air pressure regulator.

Where a different level of air filtration is required, this requirement must be agreed upon with the supplier and specified in the order.

Type	Use	Level of filtration (µm)	Article number
FS 30F		1	604014119-014
FS 30M	DK50 4VR/50	1+0.1	604014119-015
FS 30S	DK50 4VR/50/M	1+0.01	604014119-017
FS 30A		1+0.01+A**	604014119-018
FS 40F		1	604014119-000
FS 40M	DK50 2x4VR/110	1+0.1	604014119-004
FS 40S	DK50 2x4VR/110/M	1+0.01	604014119-024
FS 40AH		1+AC+HC (0.01)	604014119-005

**) A – submicrofilter with activated carbon

Filter set regulator assembly

The compressor may be equipped with a pressure regulator of the compressed air outlet if specified. The regulator must be selected according to the application to the filter set, or separately. The regulator shall ensure constant pressure at the outlet.

Type	Use	Article number
Regulator complete (to the filter assembly)	DK50 4VR/50	604014130-000
REG12 (separate)	DK50 4VR/50/M	447000001-088
Regulator complete	DK50 2x4VR/110	604014125-000
REG15	DK50 2x4VR/110/M	447000001-115

Filter set brackets

A suitable bracket must be ordered for every filter set.

Type	Use	Article number
Compressor-mounted bracket	DK50 4VR/50 DK50 4VR/50/M	603014177-000
Compressor-mounted bracket	DK50 2x4VR/110	604014131-000
Wall-mounted bracket	DK50 2x4VR/110/M	603014120-000

Equipotential bonding socket

The socket allows an equipotential bonding.

Type	Use	Name	Article number
POAG-KBT6-EC	DK50 4VR/50 DK50 4VR/50/M	Mains plug connector	033400075-000
FLEXI-S/POAG-HK6	DK50 2x4VR/110 DK50 2x4VR/110/M	Conductor (1 m)	034110083-000

9. PRODUCT FUNCTION

9.1. Compressor

The air pump (1) draws in atmospheric air through an inlet filter (8) and compresses it through a non-return valve (3) into an air tank (2) from which the device draws compressed air. If the pressure in the air tank drops to the switch-on pressure, the pressure switch (4) turns on the compressor and the compressor supplies compressed air into the air tank until it reaches the switch-off pressure, when the compressor switches off. The safety valve (5) prevents the pressure in the air tank from rising above the maximum allowed value. The drain valve (7) drains condensate from the air tank. Compressed, oil-free filtered air is stored in the air tank ready for use.

Condensate must be drained from the air tank at regular intervals (see Chapter 18.1).

9.2. Compressor with membrane dryer

The air pump (1) draws in air through an inlet filter (8) and compresses it, feeding it to the cooler (10) through the filter (18) and the non-return valve (3) into the dryer (9), and feeds the clean and dry air into the air tank (2). A part of the air is released from the dryer with the captured moisture, which is manifested as a light air stream along the body of the dryer (9). Condensate from the filter is automatically drained at regular intervals into the condensate collection vessel (12) via the condensate drain solenoid valve (17). The dryer ensures continuous drying of the compressed air. The drain valve (7) drains condensate from the air tank when drying performance is checked. Compressed, oil-free filtered air is stored in the air tank ready for use.

For the membrane dryer to work as intended, the dryer is connected directly to the air tank without the use of a check valve. As the membrane dryer contains selective membrane fibres, there is a small amount of air leakage through them (about 0.5 bar/30 min)

This is why there is a gradual, slow drop in pressure even when there is no demand for compressed air (from the air tank), however, this is not a malfunction.

The pressure vessel does not need to be drained.

9.3. Compressor cabinet

The cabinet serves as a compact soundproof compressor enclosure and ensures the sufficient exchange of cooling air. The fan under the air pump (11) and the cabinet fans (21) cool the compressor. The fans run concurrently with the compressor motor or when the temperature switch is activated at a temperature higher than 40°C. Once the space in the cabinet cools down to about 32°C, the fans turn off automatically.

Risk of compressor overheating.


Make sure that there are no obstacles at the cooling air inlet into the cabinet (around the bottom part of the cabinet) and at the hot air outlet on the top back side of the cabinet.

If the compressor is placed on a soft floor, e.g. carpet, create space between the base and the floor or the cabinet and the floor, e.g. underlay the footings with hard pads to ensure sufficient cooling of the compressor.


Descriptions for figures 1 - 3

- | | |
|---------------------------------|------------------------------------|
| 1 Air pump | 16 Electrical box / switchboard |
| 2 Air tank | 17 Condensate drain solenoid valve |
| 3 Non-return valve | 18 Filter |
| 4 Pressure switch | 21 Cabinet fan |
| 5 Safety valve | 22 Cabinet pressure gauge |
| 6 Pressure gauge | 23 Switch |
| 7 Drain valve | 24 Cabinet connector |
| 8 Inlet filter | 25 Pressure gauge hose |
| 9 Dryer | 26 Pulling system |
| 10 Cooler | 27 Wall stop |
| 11 Compressor fan | 28 Power cord |
| 12 Condensate collection vessel | 29 Cabinet gas springs |
| 13 Magnetic holder | 30 Connecting reinforcement |
| 14 Compressed air outlet | 31 Maintenance indicator |
| 15 Compressor wheel | |

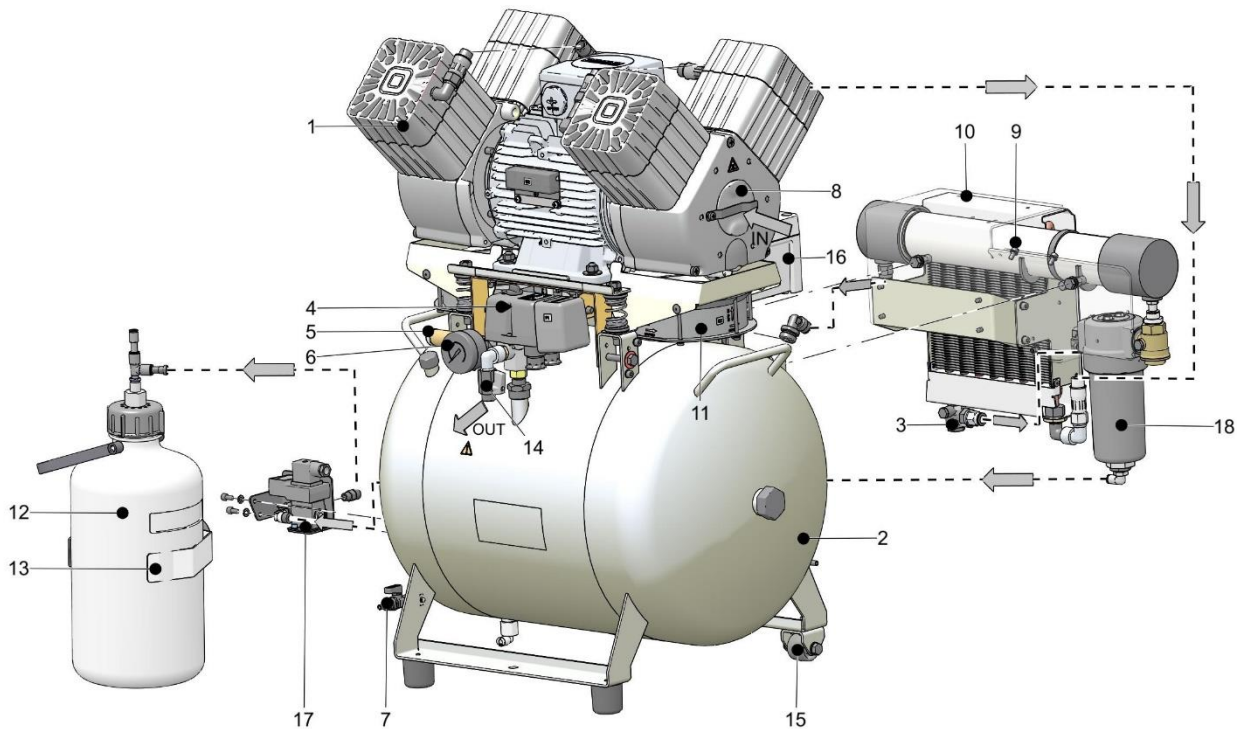Fig. 1: Compressor with membrane dryer


DK50 4VR/50/M

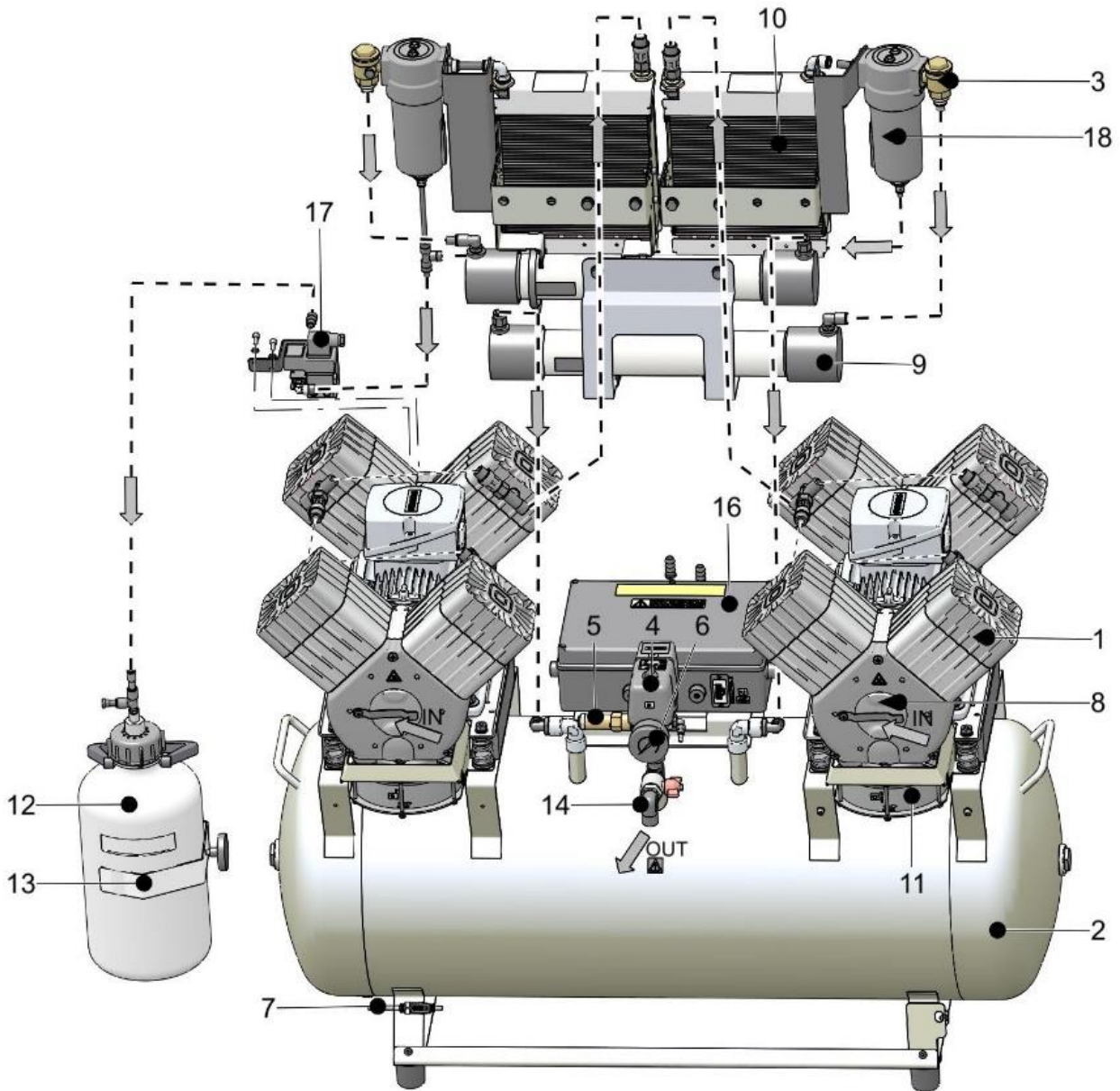
DK50 2x4VR/110/M

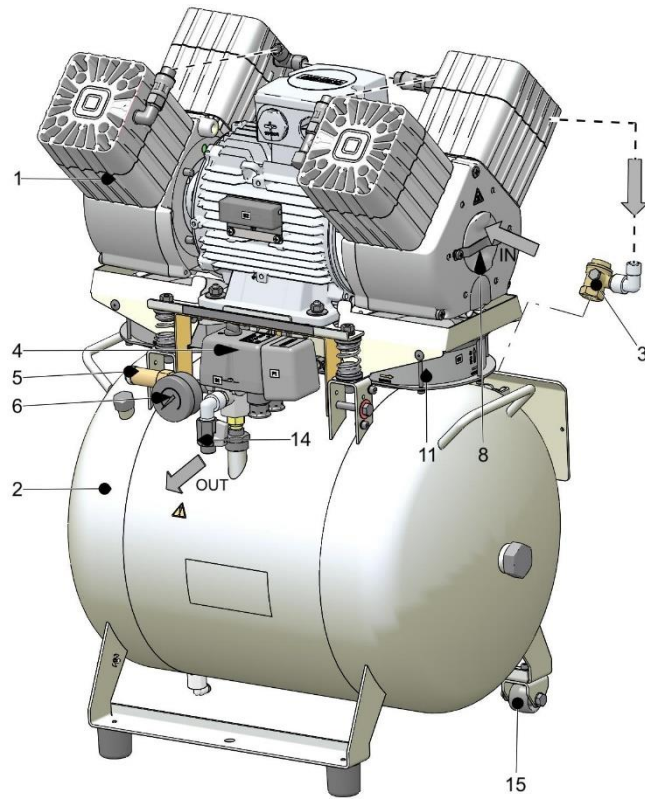Fig. 2: Compressor



DK50 4VR/50

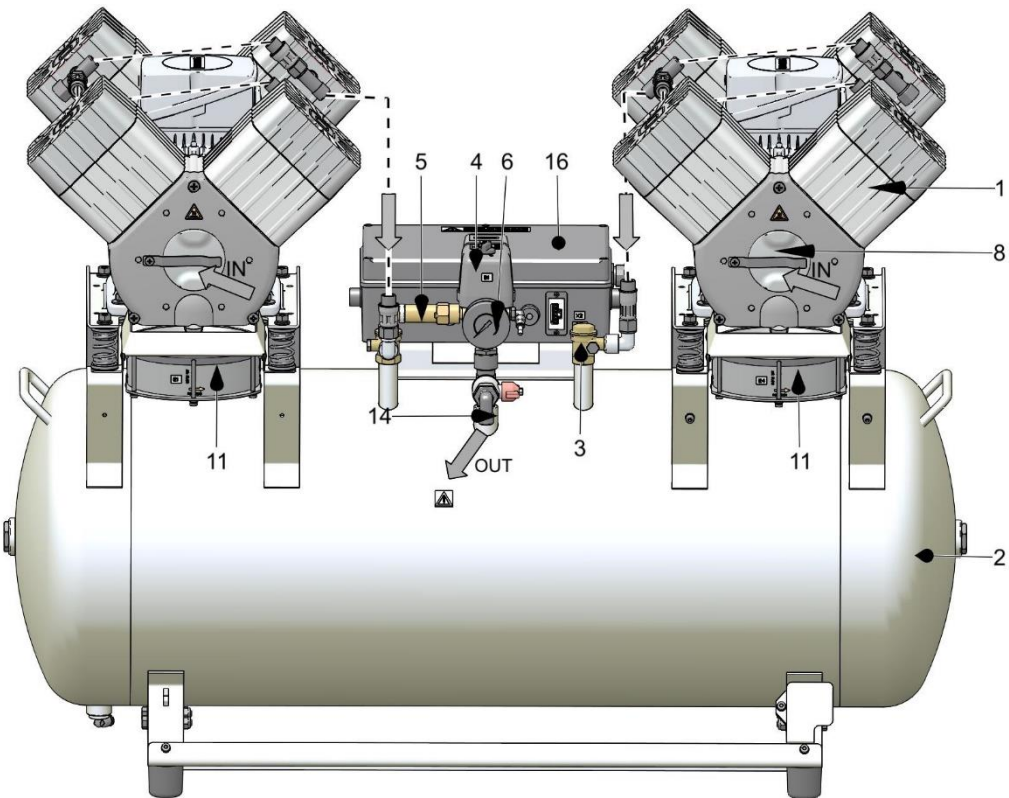
DK50 2x4VR/110

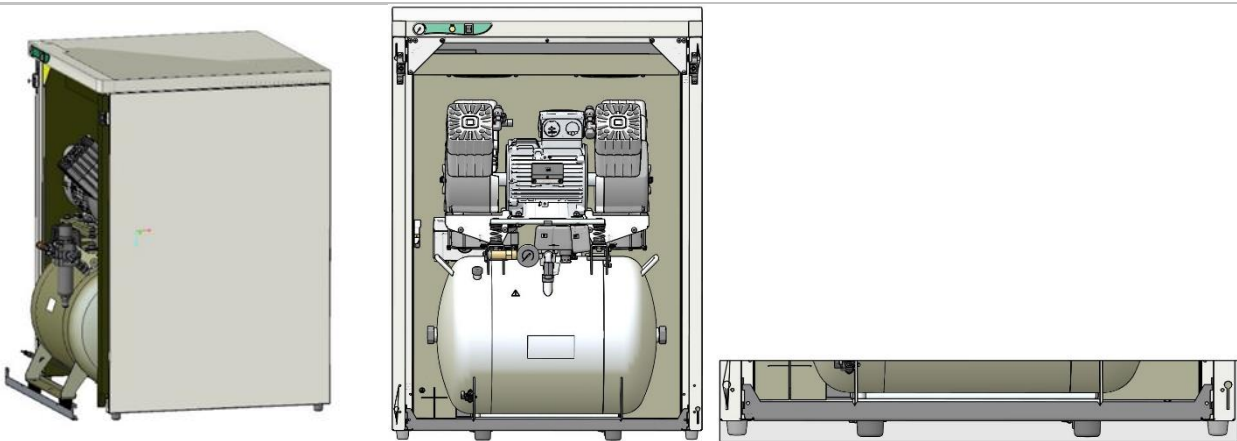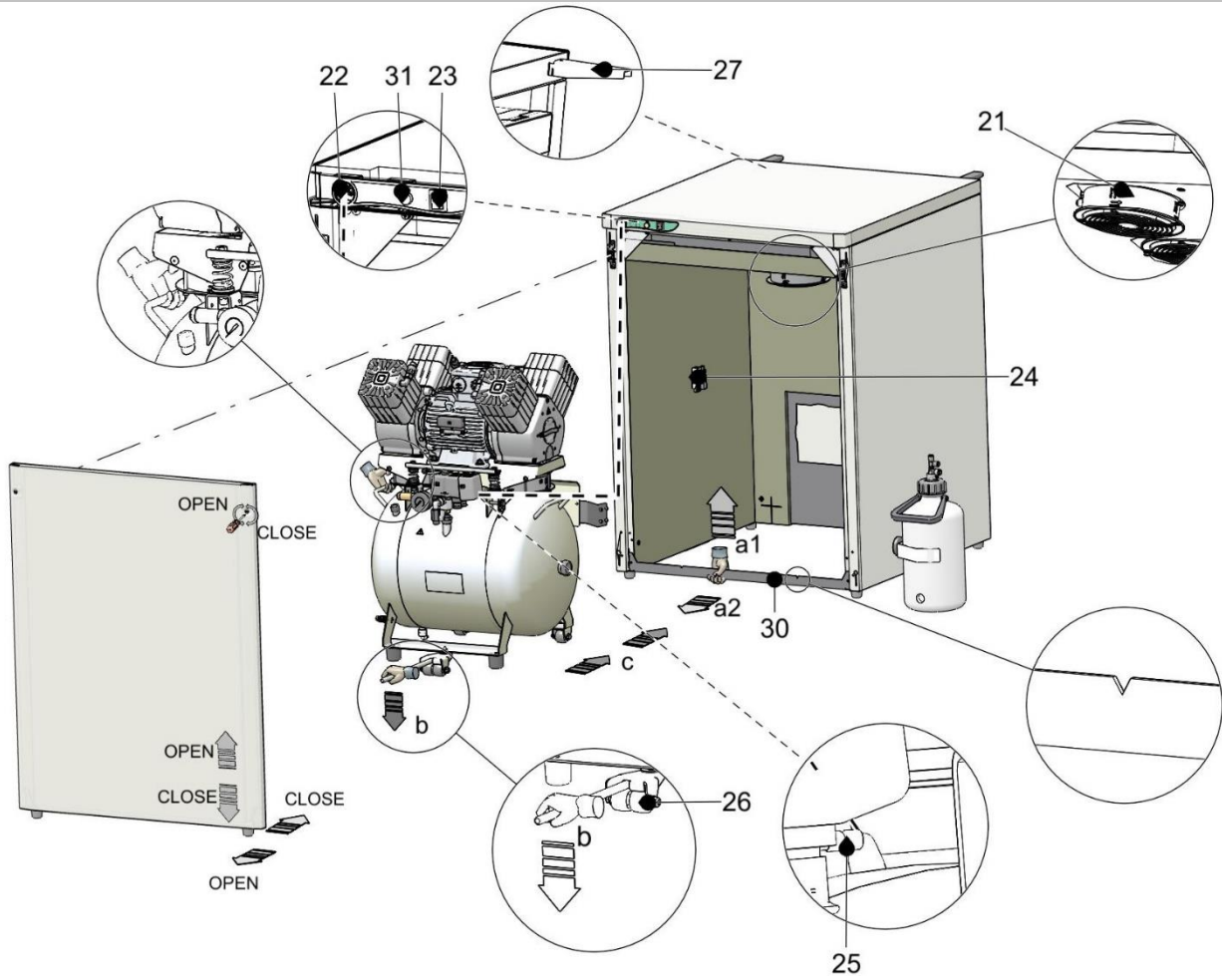Fig. 3: Cabinet

A - DK50 4VR/50S

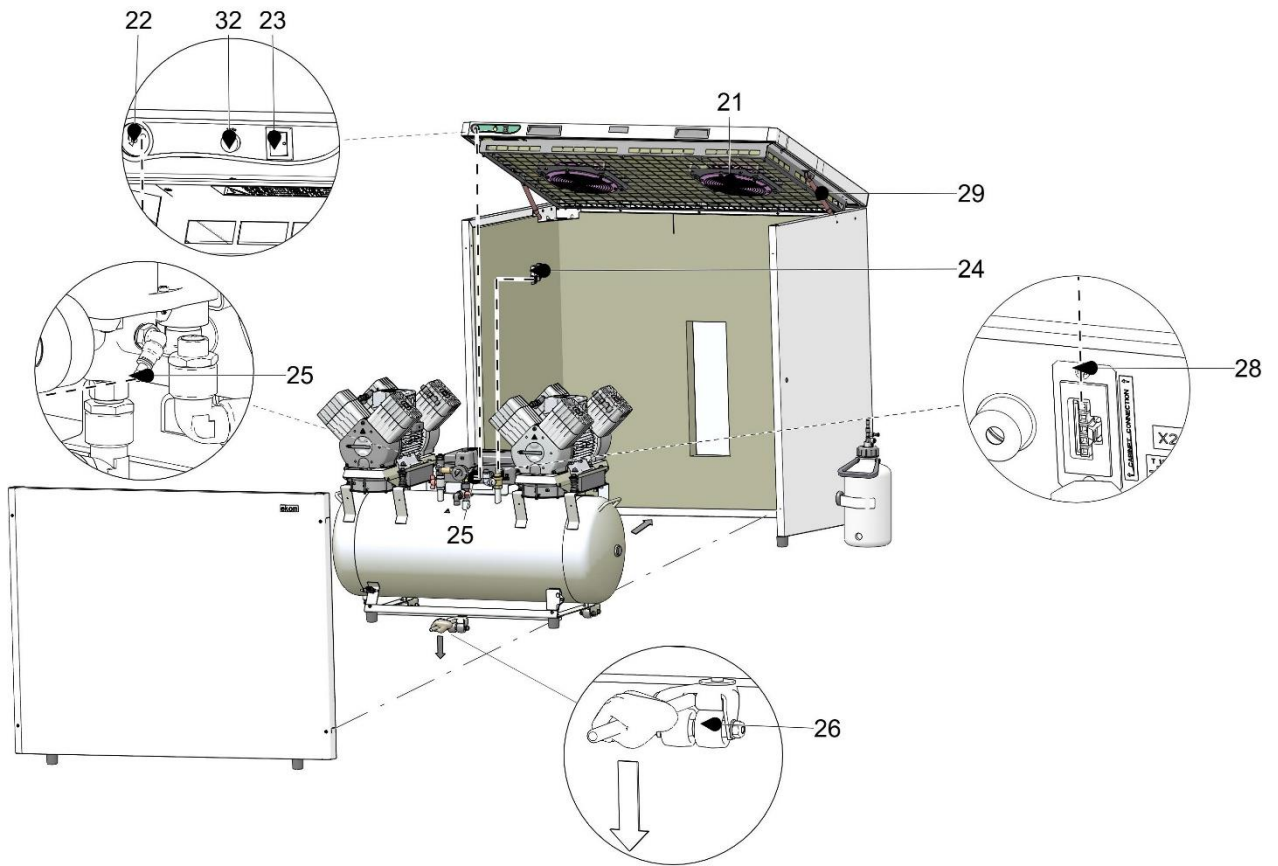B - DK50 2x4VR/110S

TECHNICAL DATA

Compressors are designed for operation in dry, ventilated and dust-free indoor rooms under the following climactic conditions:

Temperature +5°C to +40°C

Relative humidity max. 70%



Working pressure 6 – 8 bar		DK50 4VR/50	DK50 4VR/50S	DK50 4VR/50/M	DK50 4VR/50S/M
Rated voltage Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50	3x400, 50	3x400, 50
Capacity at 6 bar (FAD)	l/min	270	270	215	215
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0
Rated current	A	4.7	5.1	4.8	5.2
Motor power	kW	2.2	2.2	2.2	2.2
Air tank volume	l	50	50	50	50
Air quality - filtration	µm	-	-	0.1	0.1
Maximum operating pressure of safety valve	bar	12.0	12.0	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤77.0	≤58.0	≤77.0	≤58.0
Operating mode	%	S1-100	S1-100	S1-100	S1-100
PDP drying performance at 7 bar	°C	-	-	≤ +3	≤ +3
Dimensions (net) w x d x h	mm	577x587x815	750x778x1015	577x628x815	750x778x1015
Net weight ^{c)}	kg	70	128	81	139
Classification under EN 60601-1	Class I.				

^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

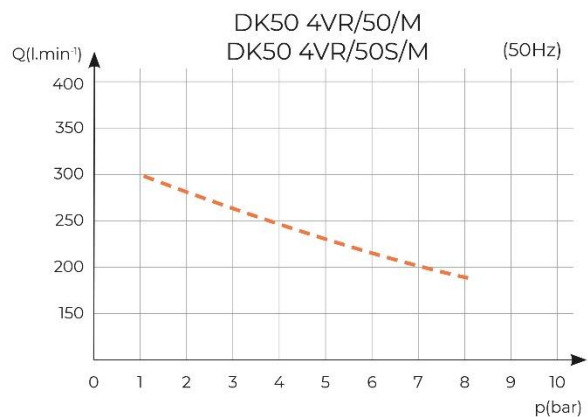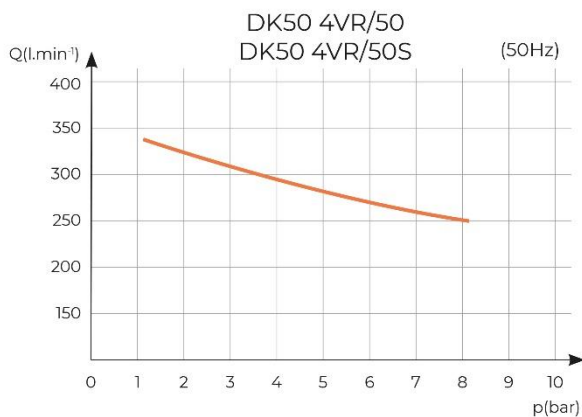Dependence of compressor output on working pressure



Working pressure 8 – 10 bar		DK50 4VR/50	DK50 4VR/50S	DK50 4VR/50/M	DK50 4VR/50S/M
Rated voltage Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50	3x400, 50	3x400, 50
Capacity at 8 bar (FAD)	l/min	220	220	175	175
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0
Rated current	A	4.7	5.1	5.4	5.9
Motor power	kW	2.2	2.2	2.2	2.2
Air tank volume	l	50	50	50	50
Air quality - filtration	µm	-	-	0.1	0.1
Maximum operating pressure of safety valve	bar	12.0	12.0	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤77.0	≤58.0	≤77.0	≤58.0
Operating mode	%	S1-100	S1-100	S1-100	S1-100
PDP drying performance at 7 bar	°C	-	-	≤ +3	≤ +3
Dimensions (net) w x d x h	mm	577x587x815	750x778x1015	577x628x815	750x778x1015
Net weight ^{c)}	kg	70	128	81	139
Classification under EN 60601-1		Class I.			

^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

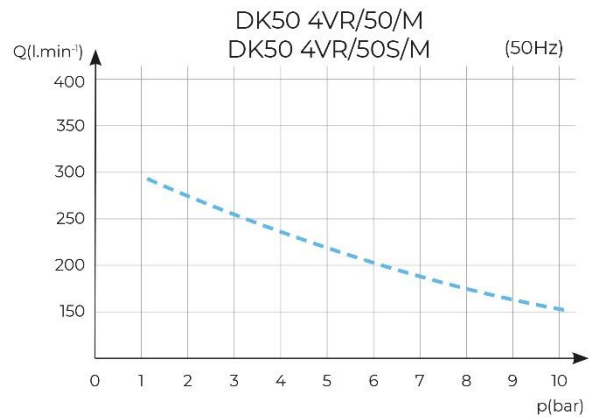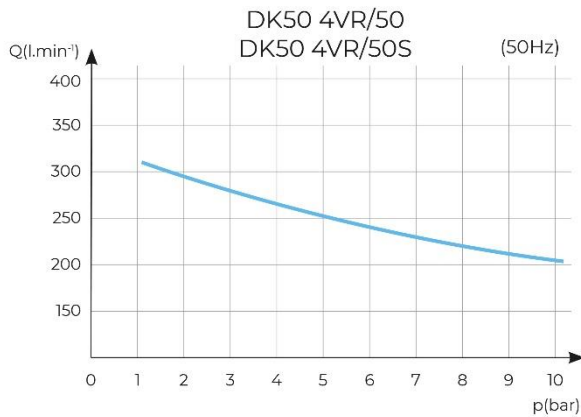Dependence of compressor output on working pressure



Working pressure 6 – 8 bar		DK50 2x4VR/110	DK50 2x4VR/110S	DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50	3x400, 50	3x400, 50
Capacity at 6 bar (FAD)	l/min	540	540	430	430
Working pressure ^{b)}	bar	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0	6.0 – 8.0
Rated current	A	9.4	10.2	10.1	10.6
Motor power	kW	2x2.2	2x2.2	2x2.2	2x2.2
Air tank volume	l	110	110	110	110
Air quality - filtration	µm	-	-	0.1	0.1
Maximum operating pressure of safety valve	bar	9.0	9.0	9.0	9.0
Noise level at 5 bar (L _{pA})	dB	≤77.0	≤58.0	≤77.0	≤58.0
Operating mode	%	S1-100	S1-100	S1-100	S1-100
PDP drying performance at 7 bar	°C	-	-	≤ +3	≤ +3
Dimensions (net) w x d x h	mm	1085x511x830	1260x834x1046	1085x678x830	1260x834x1046
Net weight ^{c)}	kg	136	218	158	240
Classification under EN 60601-1	Class I.				

^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

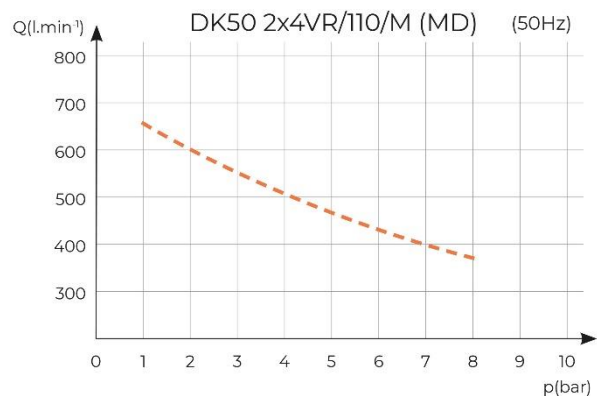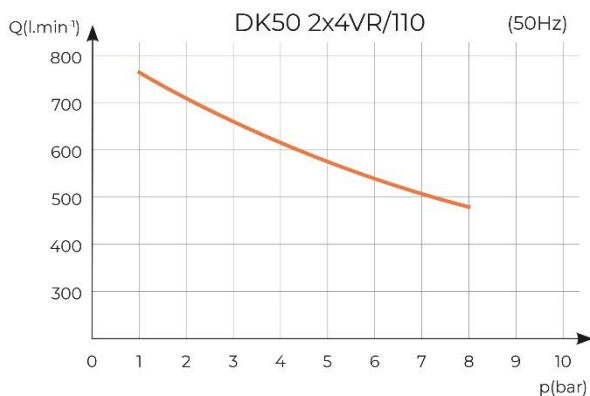Dependence of compressor output on working pressure



Working pressure 8 – 10 bar		DK50 2x4VR/110	DK50 2x4VR/110S	DK50 2x4VR/110/M	DK50 2x4VR/110S/M
Rated voltage Frequency ^{a)}	V, Hz	3x400, 50	3x400, 50	3x400, 50	3x400, 50
Capacity at 8 bar (FAD)	l/min	440	440	350	350
Working pressure ^{b)}	bar	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0	8.0 – 10.0
Rated current	A	9.4	10.2	10.1	10.6
Motor power	kW	2x2.2	2x2.2	2x2.2	2x2.2
Air tank volume	l	110	110	110	110
Air quality - filtration	µm	-	-	0.1	0.1
Maximum operating pressure of safety valve	bar	12.0	12.0	12.0	12.0
Noise level at 5 bar (L _{pA})	dB	≤77.0	≤58.0	≤77.0	≤58.0
Operating mode	%	S1-100	S1-100	S1-100	S1-100
PDP drying performance at 7 bar	°C	-	-	≤ +3	≤ +3
Dimensions (net) w x d x h	mm	1085x511x830	1260x834x1046	1085x678x830	1260x834x1046
Net weight ^{c)}	kg	136	218	158	240
Classification under EN 60601-1	Class I.				

^{a)} Specify the compressor version when ordering

^{b)} Consult any other range of pressure with the supplier

^{c)} Weight is indicative and only applies to the product without accessories

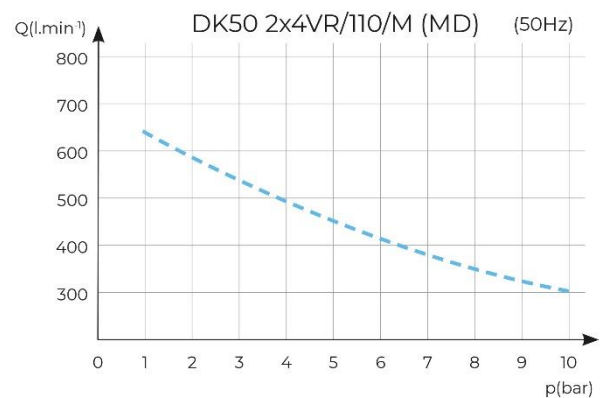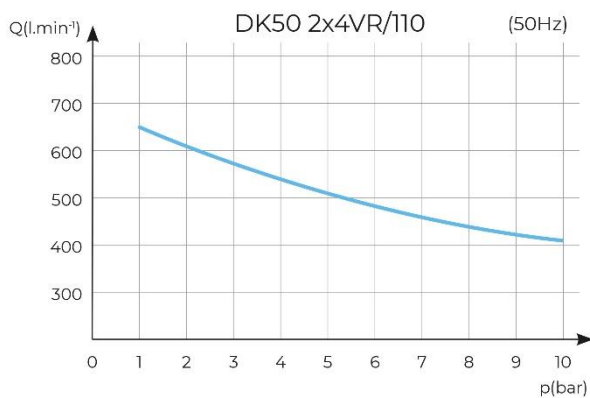Dependence of compressor output on working pressure

FAD correction of capacity for altitude

Capacity given in the form of FAD („Free Air Delivery“) applies to the following conditions:

Altitude	0 m.n.m.	Temperature	20°C
Atmospheric pressure	101325 Pa	Relative humidity	0%

To calculate FAD compressor capacity in dependence on altitude, it is necessary to apply correction factor according to the following table:

Altitude [m.n.m.]	0 -1500	1501 - 2500	2501 - 3500	3501 - 4500
FAD correction factor	1	0.80	0.71	0.60

Electromagnetic compatibility declaration

Medical equipment needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided below.

Guidance and manufacturer's declaration - electromagnetic emissions		
Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances		
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal functions. Therefore, the RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	The equipment is not likely to cause any flicker, as the current flow is approx. constant after the start up.	

Guidance and manufacturer's declaration - electromagnetic immunity			
Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances			
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV 100 kHz repetition frequency Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV L-N ±2 kV L-PE; N-PE Applied to mains connection	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruption, and voltage variations on power supply input lines IEC 60601-4-11	$U_T=0\%$, 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T=0\%$, 1 cycle $U_T=70\%$ 25/30 cycles (at 0°) $U_T=0\%$, 250/300 cycles	$U_T=>95\%$, 0,5 cycle (at 0, 45, 90, 135, 180, 225, 270 and 315°) $U_T=>95\%$, 1 cycle $U_T=70\%$ (30% dipp U_T), 25(50Hz)/30(60Hz) cycles (at 0°) $U_T=>95\%$, 250(50Hz)/300(60Hz) cycle	Mains power quality should be that of a typical commercial or hospital environment. The device stops and restarts automatically at each dip. In this case does not occur unacceptable pressure drop.
Power frequency (50/60 Hz) IEC 61000-4-8	N/A	N/A	The equipment does not contain magnetically sensitive components or circuits.
NOTE: U_T is the A.C. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration - electromagnetic immunity			
Pursuant to IEC 60601-1-2:2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances			
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80MHz	3 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the equipment, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d=1,2\sqrt{P}$ $d=1,2\sqrt{P}, 80 \text{ MHz to } 800 \text{ MHz}$ $d=2,3\sqrt{P}, 800 \text{ MHz to } 2,7 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a, should be less than the compliance level in each frequency range ^b. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,7 GHz	3 V/m	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	9 to 28 V/m 15 specific frequencies (380 to 5800 MHz)	
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p>^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the equipment.</p> <p>^b Over the frequency range 150 kHz to 80 kHz, field strengths should be less than 3 V/m.</p>			

INSTALLATION

Risk of incorrect installation.

Only a qualified technician may install the compressor and place it into operation for the first time. Their duty is to train operating personnel on the use and maintenance of the equipment. An entry is made in the equipment installation record to certify installation and operator training. (see the Annex chapter)

10. INSTALLATION CONDITIONS

- The compressor may only be installed and operating in dry, well-ventilated and clean environments under the conditions specified in the Technical Data chapter.

Risk of damage to the device.

The equipment may not be operated outdoors or in otherwise wet or damp environments.

Risk of explosion.

Do not use the equipment in the presence of explosive gases, dust or combustible liquids.

Burn or fire hazard! Caution! Hot surface!

Portions of the air pump may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.

- The compressor must be installed so that it is accessible at all times for operating and maintenance. Please ensure that the nameplate on the device is readily accessible.
- The compressor must stand on a flat, sufficiently stable base (be aware of the weight of the compressor, see the Technical Data chapter).
- Any other use or use outside this framework is not considered as intended use. The manufacturer is not responsible for any damages arising from such use. The operator/user assumes all risks

High temperature hazard.

The placement of air flow impediments upstream or downstream of the cooler is prohibited. The temperature of internal and external parts of the cooler may be hot and reach hazardous temperatures.

You may notice a “new product” odour when you first place the product into service (for a short period of time). This odour is temporary and does not impede the normal use of the product. Ensure the space is properly ventilated after installation.

11. COMPRESSOR ASSEMBLY

11.1. Handling and releasing the compressor

- Unpack the compressor from the packaging.
- Position the compressor at the site of installation. (Fig. 4)


Use the handles on the compressor to move the compressor as needed. Do not use other parts of the compressor (the air pump, etc.) for grip


At least two persons are needed to handle the equipment.

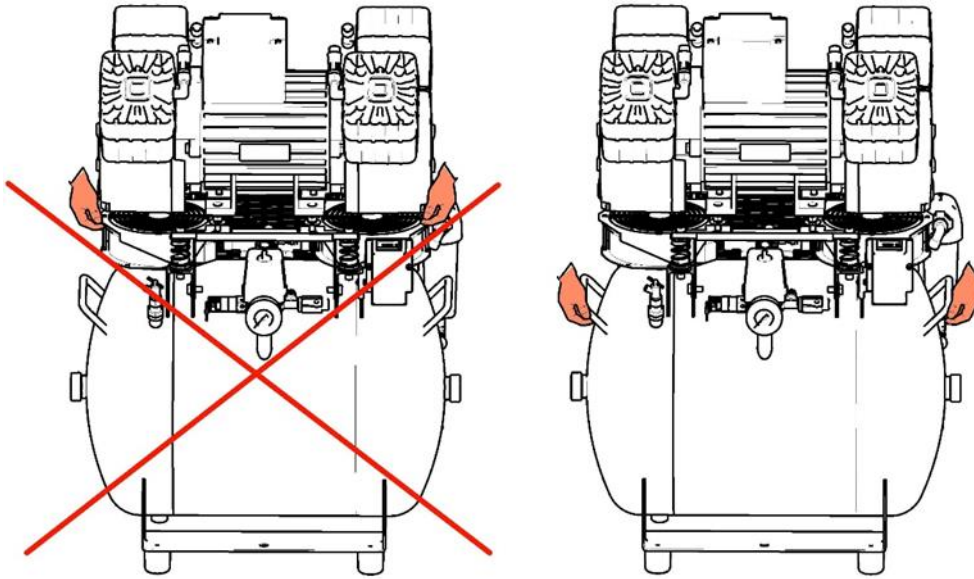Fig. 4: Handling the compressor

DK50 4VR/50

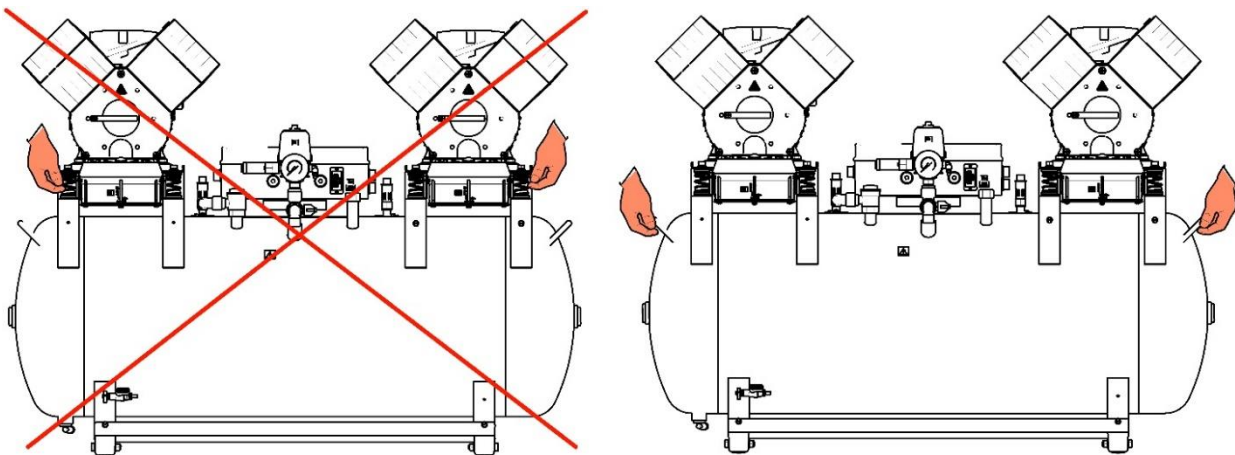DK50 2x4VR/110


- Remove the transport stabilisers from the air pumps (Fig. 5).

Prior to installation, ensure that the compressor is free of all transport packaging and stabilizers to avoid any risk of damage to the product.

Remove all devices used to secure the aggregates once the compressor is installed and levelled at the site of final installation.

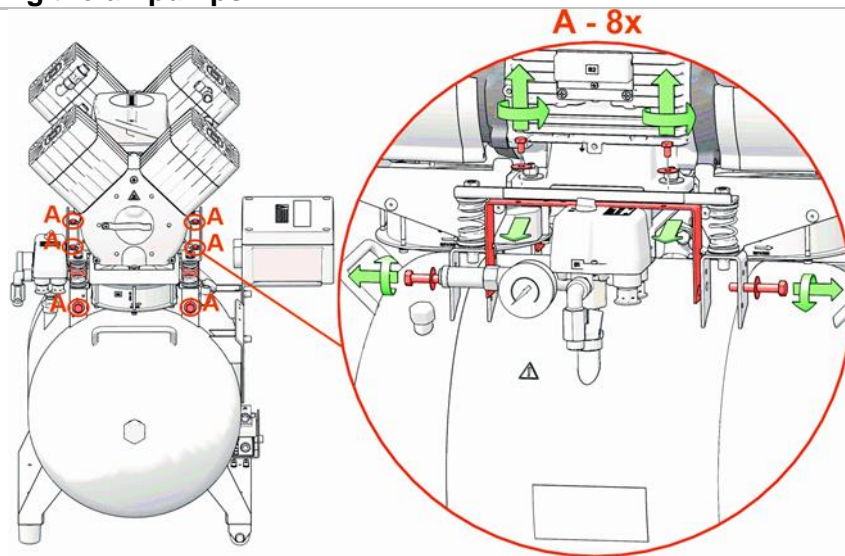Fig. 5: Releasing the air pumps

DK50 4VR/50

DK50 2x4VR/110

11.2. Placement of the compressor in the cabinet

DK50 4VR/50S (Fig. 3-A):

Disassembly cabinet door

- Remove the door held by 2 screws and disconnect the earthing lead.

Compressor placement

- Remove the connecting strip (30) in front of the cabinet.
- Insert the compressor into the cabinet using the handle (26), place the connecting strip in front of the cabinet and adjust the position of the compressor so that the two “V” notches on the connecting strip (30) align with the compressor legs. This puts the compressor in the right position inside the cabinet.
- Insert the compressor with dryer into the cabinet so that the dryer fan fits into the cooling tunnel opening in the cabinet. **WARNING**, do not push the compressor all the way against the stop. For dryer versions of the compressor, the flow of cooling air into the cooler will be restricted and it may permanently damage the dryer.

- Place the connecting strip (30) in its original position in the lower part of the cabinet.

Cabinet door installation

- Align the door with the cabinet, connect the earthing lead to the door and mount the door to the cabinet using 2 fasteners.

Do not push the compressor all the way to the back of the cabinet as it may permanently damage the dryer.

DK50 2x4VR/110S (Fig. 3-B):

Opening the upper cover

- Open the lock by turning with a screwdriver as shown in the pictogram and pick up using the handle.
- The gas springs will keep the cover open.

Be careful not to pinch your fingers when closing the cover on the cabinet.

Always lock with the quick releases after closing the cover on the cabinet!

Removal of the front panel on the cabinet

- Remove the door held by 4 (M5) fasteners and disconnect the earthing lead.

Compressor placement

- Insert the compressor with dryer into the cabinet so that the dryer fan fits into the cooling tunnel opening in the cabinet. Check the positioning of the compressor in the cabinet against Fig. 6

Do not push the compressor all the way to the back of the cabinet as it may permanently damage the dryer.

Cabinet door installation

- Attach the door to the cabinet and connect the earthing lead to the door. Install the door on the cabinet and mount using the 4 fasteners.

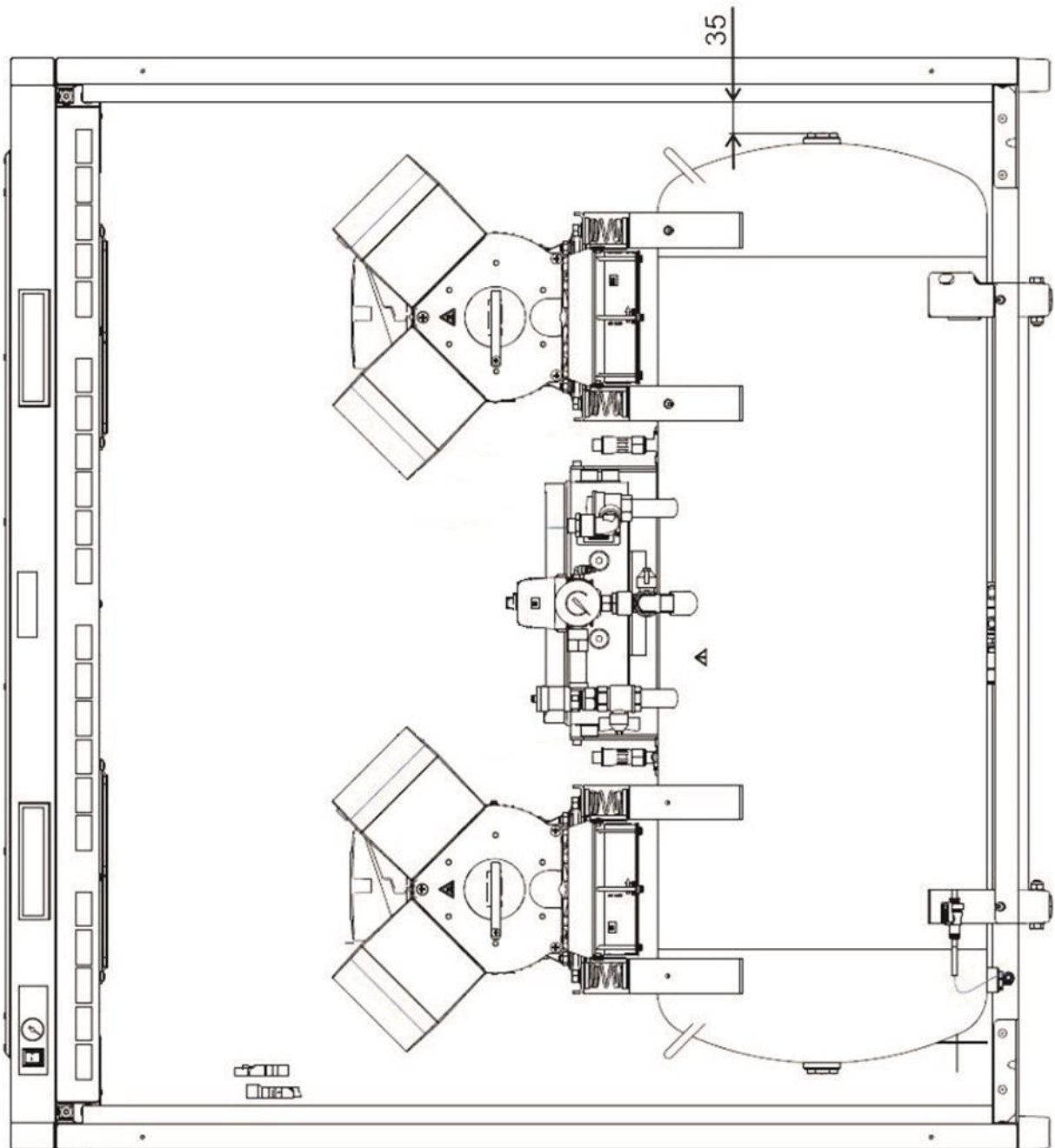
Fig. 6: Placement of the compressor in the cabinet

11.3. Valve installation on the condensate drain from the cabinet (DK50 2x4VR/110)

For cabinet-mounted compressors, install the threaded fitting with the valve (1) into the hole in the side of the cabinet and install the PA $\text{\O} 8 / \text{\O} 6$ hose (2). Insert the other side of the hose into the fitting (4) beneath the air tank from which the valve (3) and hose are removed.

Use a $\text{\O}16.5$ plug to close off the hole on the other side of the cabinet.

The fitting with the valve are standard parts delivered with the cabinet. The condensate drain valve may be installed on either side of the compressor.

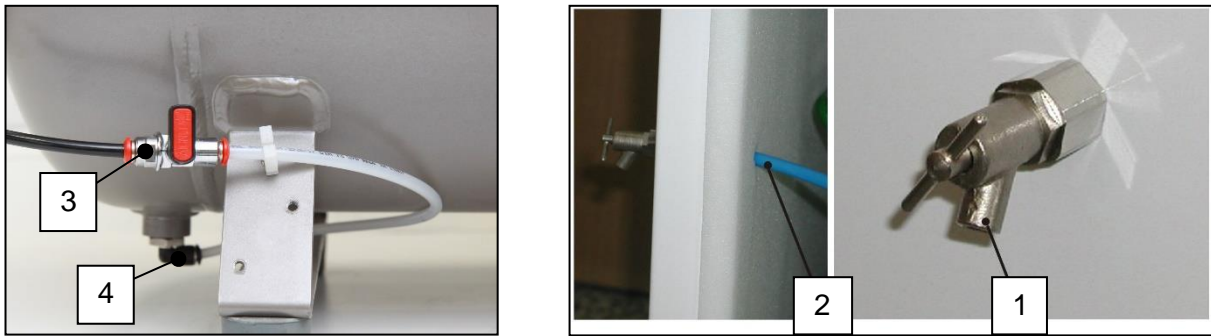
Fig. 7: Valve installation on the condensate drain from the cabinet (DK50 2x4VR/110)

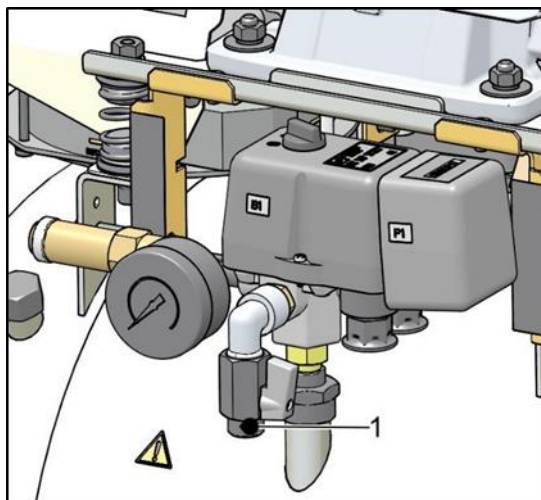
Use a screwdriver to disconnect the cabinet connector when disassembling the compressor!

12. PNEUMATIC CONNECTION

12.1. Connecting to the compressed air outlet

- Route the pressure hose from the compressed air outlet (1) to the connected equipment.
- Route the pressure hose through the opening in the rear wall of the cabinet for cabinet-mounted compressors.
- A G 3/8" (DK50 4VR/50), G1/2" (DK50 2x4VR/110) connection is installed.

DK50 4VR/50

DK50 2x4VR/110

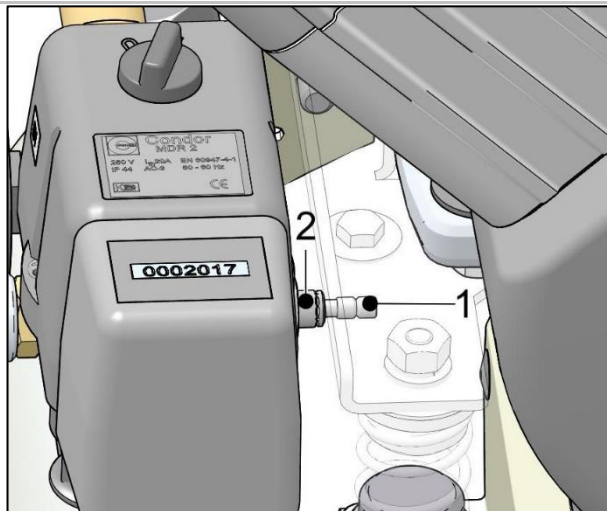
Fig. 8: Connecting to the compressed air outlet

12.2. Connecting the cabinet pressure gauge to the compressor

- Remove the plug (1) from the threads (2) on the pneumatic block of the compressor.
- Connect the cabinet pressure gauge hose to the threaded fitting.

DK50 4VR/50

DK50 2x4VR/110

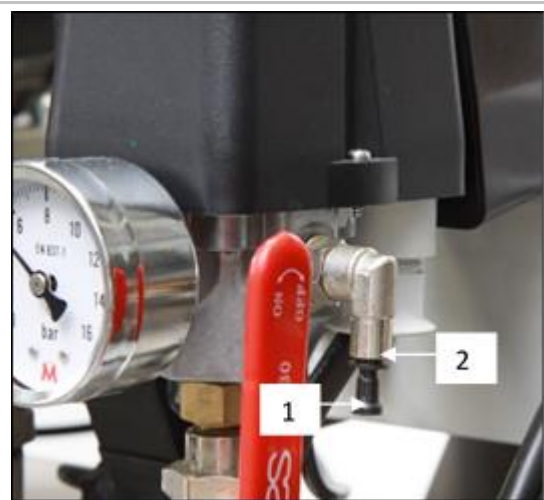
Fig. 9: Connecting the cabinet pressure gauge to the compressor

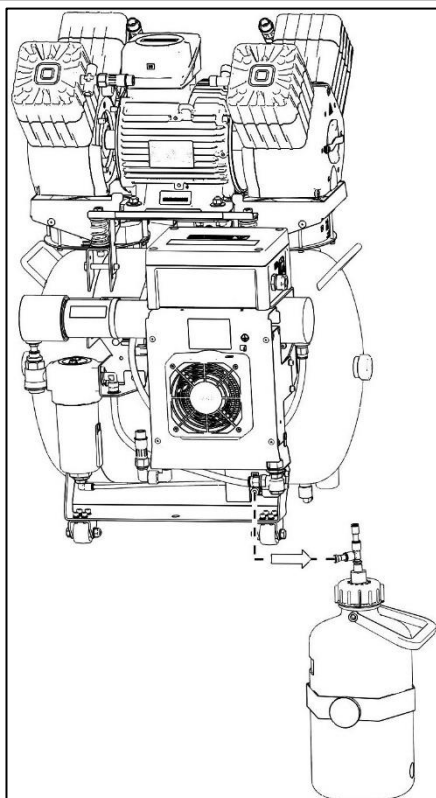
Condensate outlet from dryer

Do not connect the condensate drain directly to a waste drain! Passers-by may be injured!

- Connect a hose to the outlet from the automatic condensate drain to drain piping or to the provided condensate collection vessel.

DK50 4VR/50

DK50 2x4VR/110

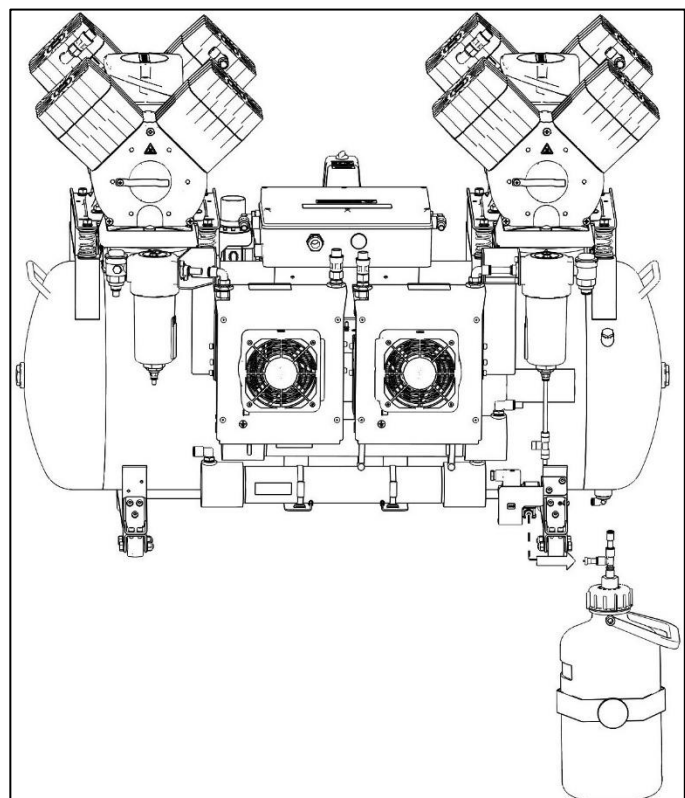
Fig. 10: Condensate drain

12.3. Routing hoses and electrical cables in cabinet-mounted compressors

- Route the hose and electrical cables through the opening in the rear wall of the cabinet for compressor models with a dryer.

Risk of damage to pneumatic components.

Air hoses must not be broken.

13. ELECTRICAL CONNECTION

- The product is delivered with a cord equipped with a plug with earthing pin.
- With cabinet-mounted compressors, route the power cord through the opening in the back wall of the cabinet.
- Keep the socket easily accessible to ensure that the device can be safely disconnected from the mains.
- Connection to the power distribution cabinet must be max.16 A

Risk of electric shock.

It is necessary to follow all local electro technical regulations. The mains voltage and frequency must comply with the data stated on the device label.

Risk of fire and electric shock.

Electrical cord must not be broken.

Risk of fire and electric shock.

Electrical cable must not be in contact with hot compressor components.

- Connect the pin for equipotential bonding 6Ø mm (1) to the electrical circuit using the defined method under valid electrical engineering regulations
- The socket for equipotential bonding (2) is an accessory and is not provided in the basic package

Fig. 11 :Equipotential bonding

- Mount the connector of the grounding conductor on the door and screw on the cabinet.

Power supply connection

Supply voltage	Electric wire type
230V/50 (60) Hz	3Gx1.5x4000
400V/50 (60) Hz	5Gx1.5x4000

13.1. Connecting a compressor not installed in a cabinet

- Insert the mains plug into a rated mains socket.
- The compressor is ready for operation.

13.2. Connecting a compressor installed in a cabinet

- Route the mains plug through the opening in the rear wall of the cabinet for cabinet-mounted compressors. (Fig. 12)
- Connect the cabinet electrically to the compressor by inserting the provided power cord with connector into an outlet. (Fig. 13)
- Disconnect the cabinet electrically from the compressor by pulling the connector from the outlet while the latch is released.

Fig. 12: Opening in the rear wall of the cabinet

Fig. 13: Connect the cabinet to the compressor

14. COMMISSIONING

- Make sure all transport stabilizers were removed.
- Check that all compressed air hose connections are correct. (see chap. 12).
- Check correct connection to the mains (see chap. 13).
- Check the circuit breaker position, it must be in the "I" position. If the main switch (2) is in the "0" position, turn it to the "I" position (Fig. 14).
- For cabinet-mounted compressors, turn the main switch (4) on the front side of the cabinet into the "I" position; a green indicator indicates the equipment is in operation. (Fig. 14).
- Check connection of the cabinet connector to the compressor (Fig. 13).

- Check connection of the cabinet manometer hose to the pneumatic block of the compressor (Fig. 9).
- Check to ensure the hose from the automatic condensate drain is connected to the condensate collection vessel. (Fig. 10).

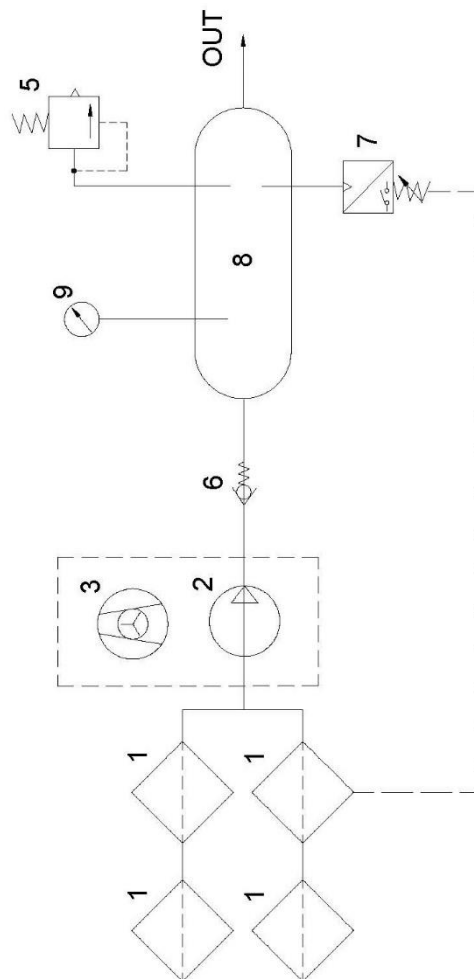
The compressor is not equipped with a backup power supply.


15. PNEUMATIC AND ELECTRICAL DIAGRAMS

15.1. Pneumatic diagram


DK50 4V/50


DK50 4V/50/M

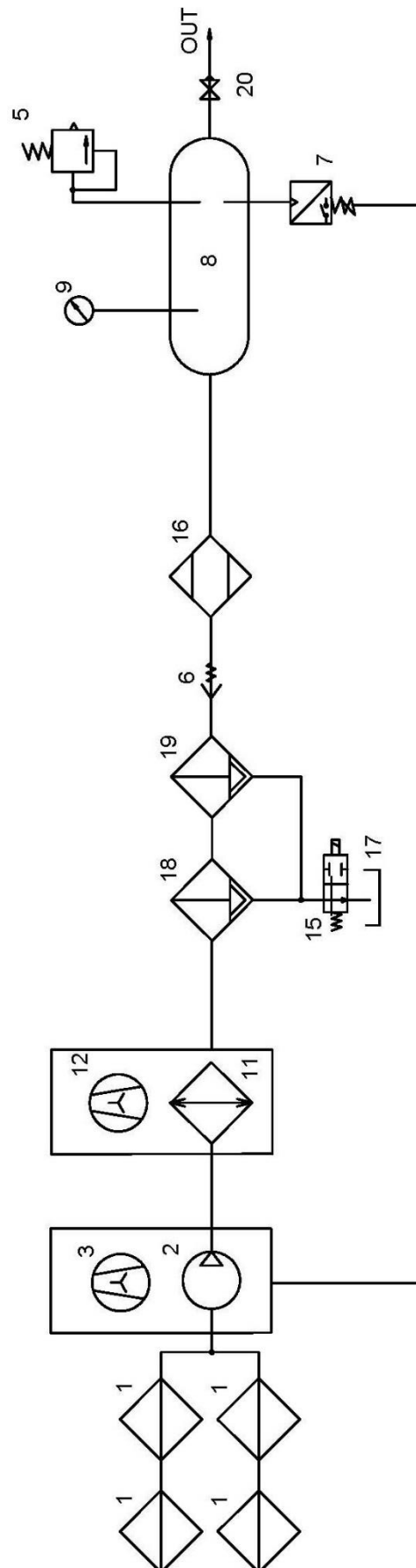DK50 2x4VR/110

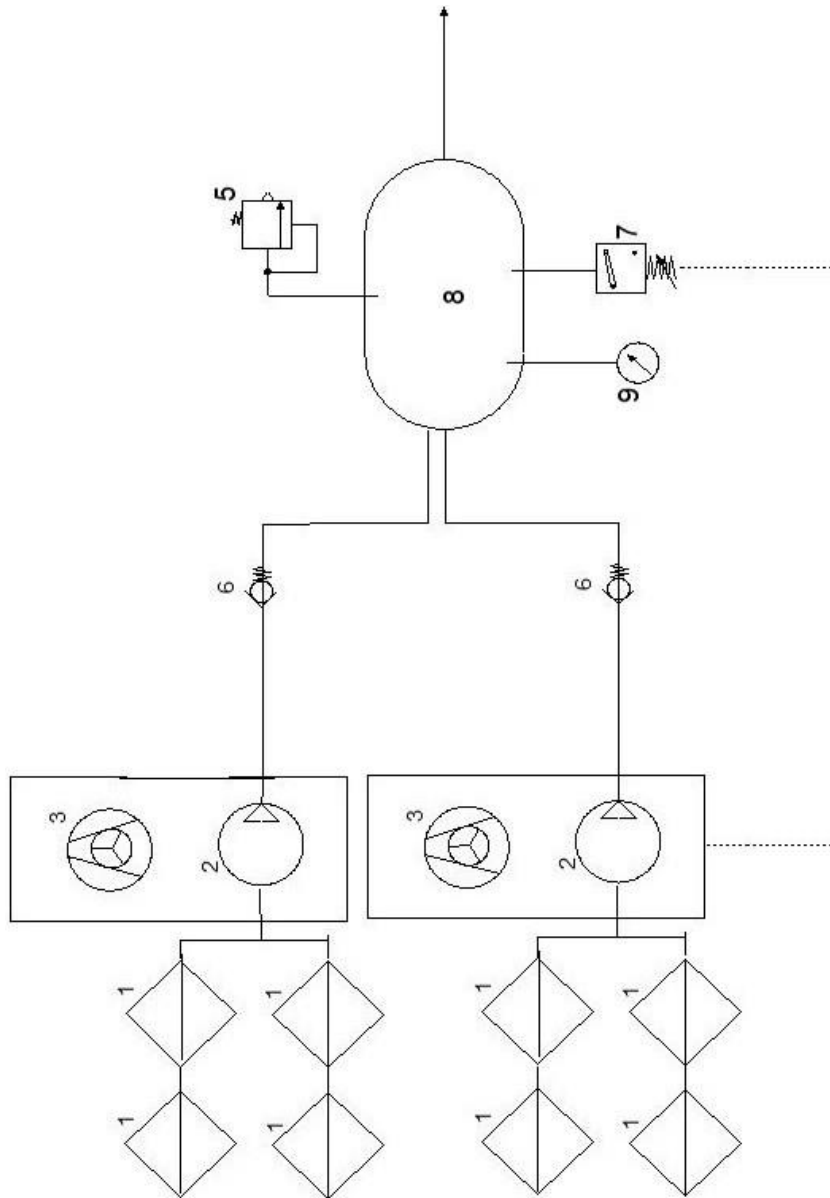DK50 2x4VR/110/M

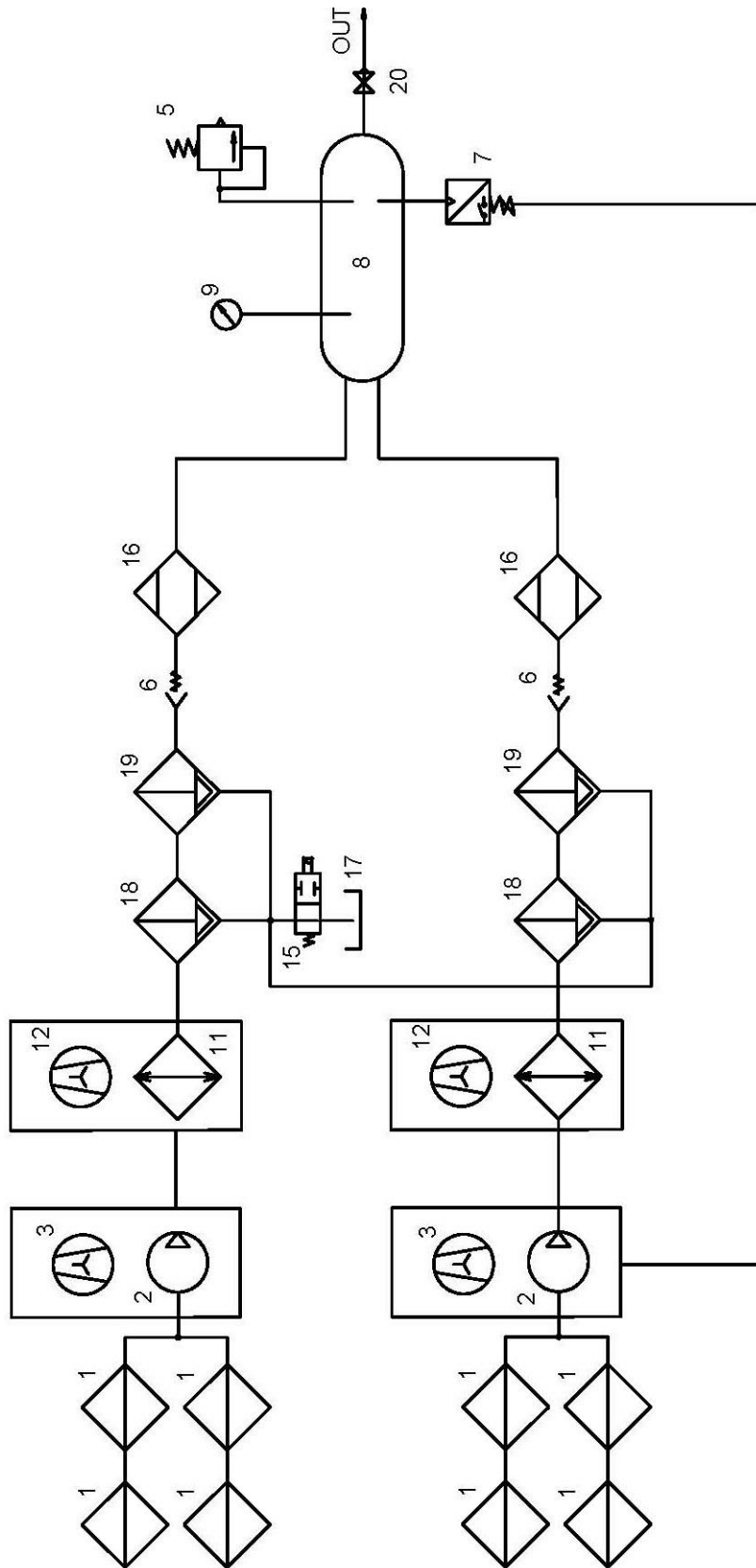Description to pneumatic diagram:

- | | |
|--------------------|------------------------------------|
| 1 Inlet filter | 11 Cooler |
| 2 Air pump | 12 Cooler fan |
| 3 Fan | 13 – |
| 4 - | 14 – |
| 5 Safety valve | 15 Condensate drain solenoid valve |
| 6 Non-return valve | 16 Dryer |
| 7 Pressure switch | 17 Condensate collection vessel |
| 8 Air tank | 18 Filter |
| 9 Pressre gauge | 19 Microfilter |
| 10 - | |

15.2. Electrical diagrams

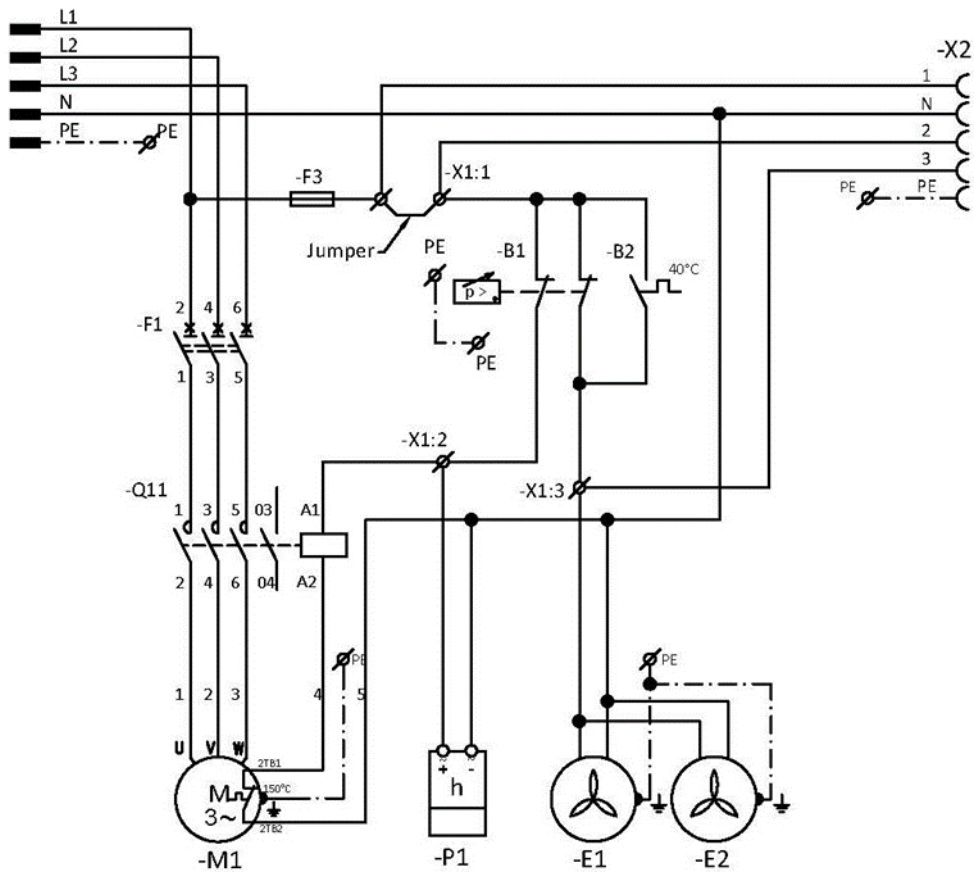
DK50 4VR/50

6 - 8 bar, 8 – 10 bar

3/N/PE-400V, 50Hz

ELEKTRIC MAIN TN-S [TN-C-S]

ELEKTRIC OBJECT OF 1st. CAT.


DK50 4VR/50/M

3/N/PE~400V, 50Hz

ELEKTRIC MAIN TN-S [TN-C-S]

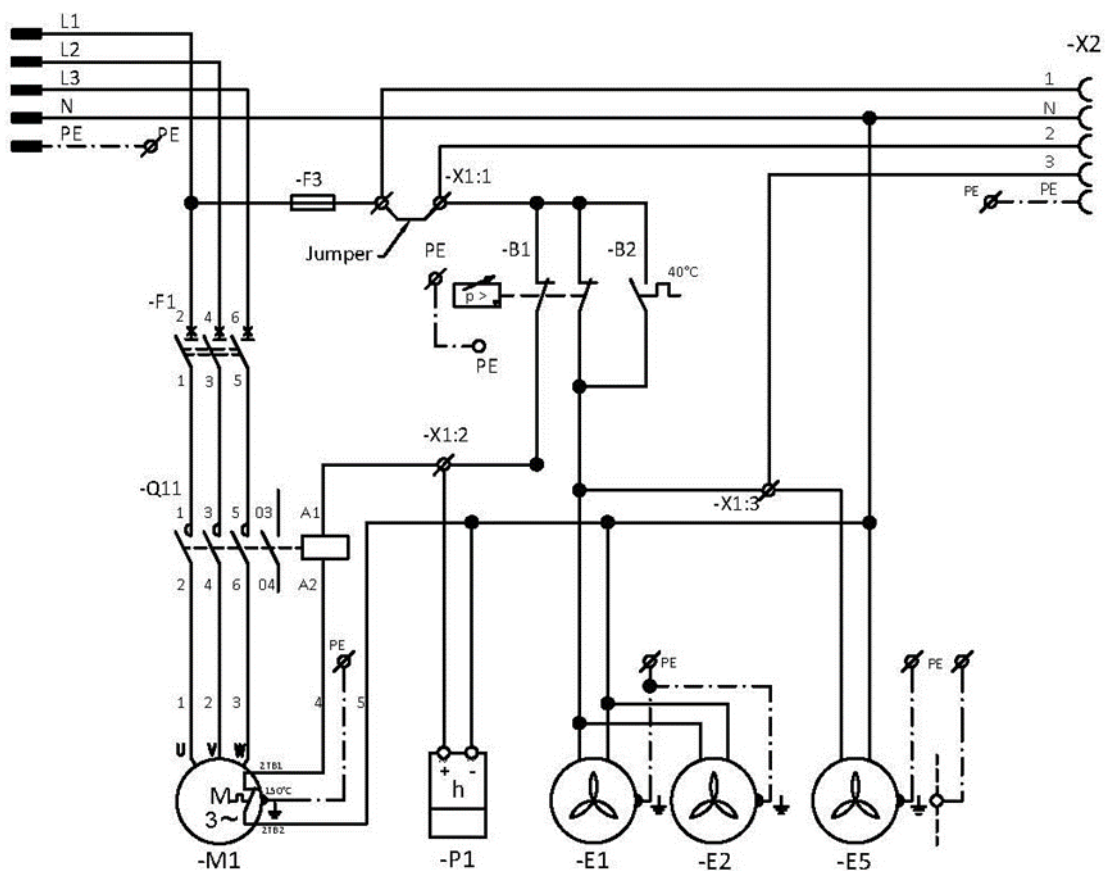ELEKTRIC OBJECT OF 1st. CAT.


DK50 2x4VR/110

6 - 8 bar, 8 - 10 bar

3/N/PE~400V, 50Hz

ELEKTRIC MAIN TN-S [TN-C-S]

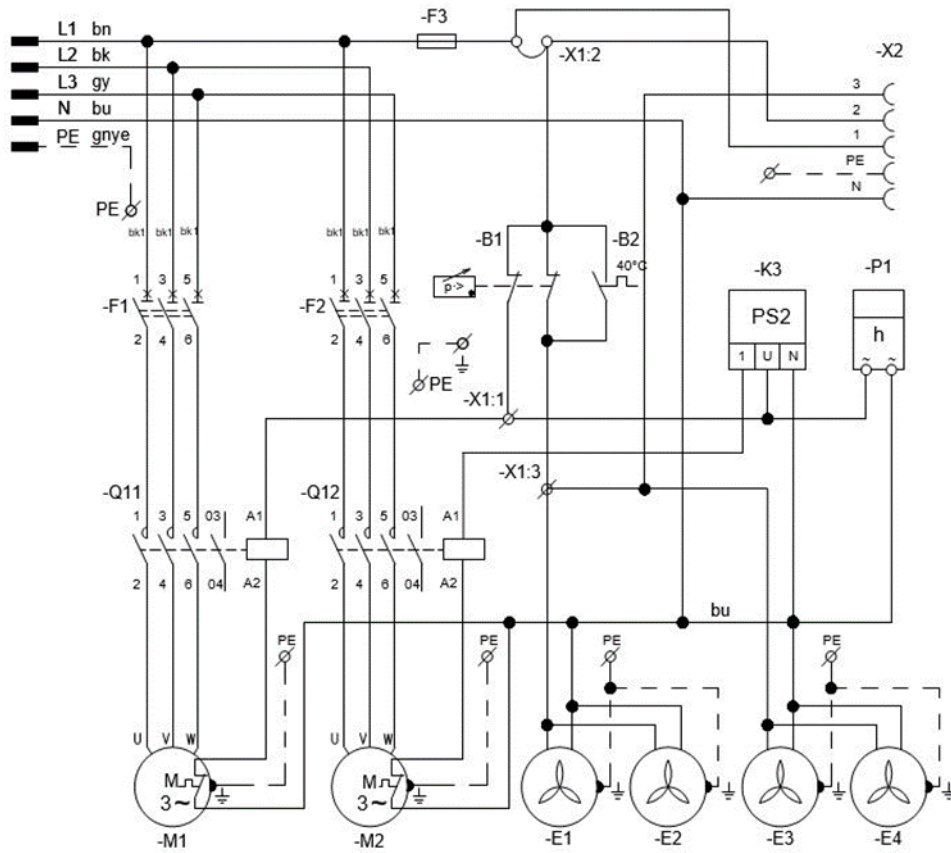ELEKTRIC OBJECT OF 1st. CAT.


DK50 2x4VR/110/M

6 - 8 bar, 8 - 10 bar

3/N/PE~400V, 50Hz


ELEKTRIC MAIN TN-S [TN-C-S]

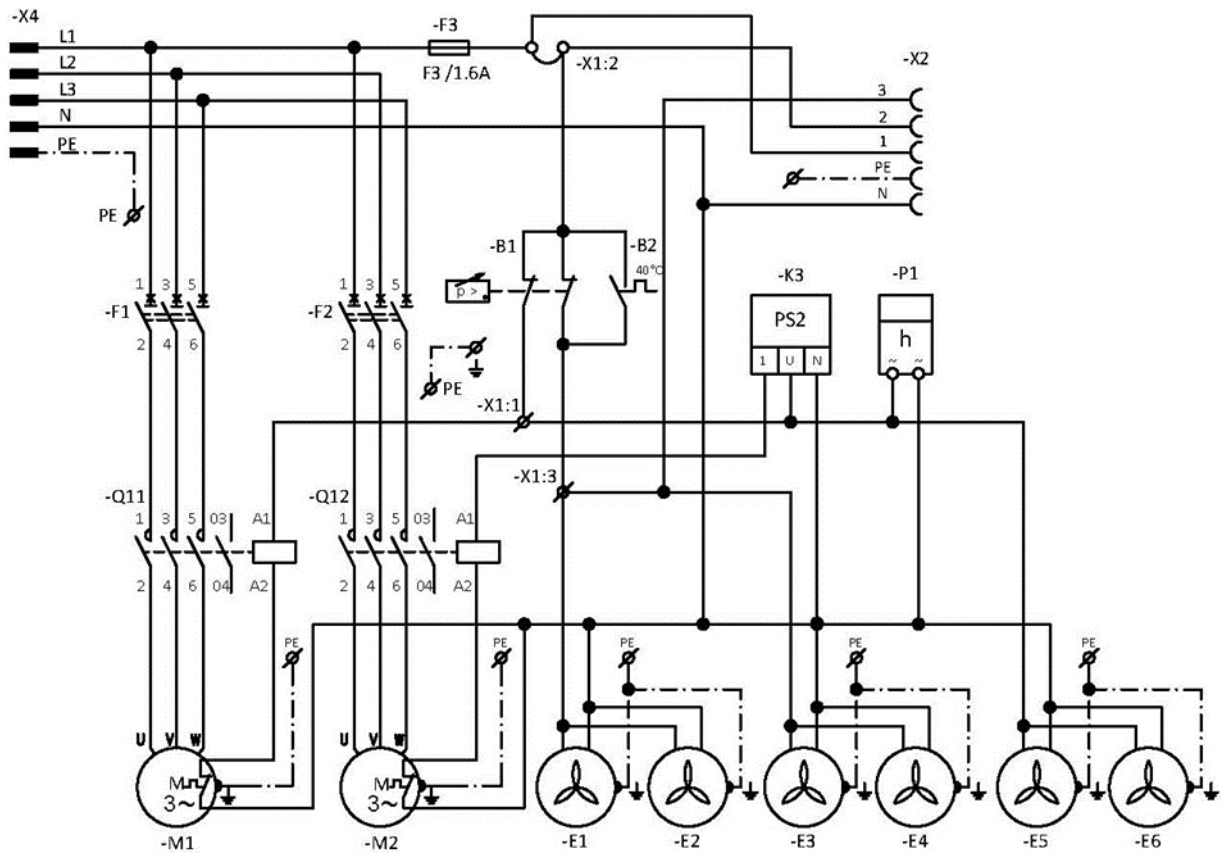ELEKTRIC OBJECT OF 1st. CAT.


Compressor cabinet

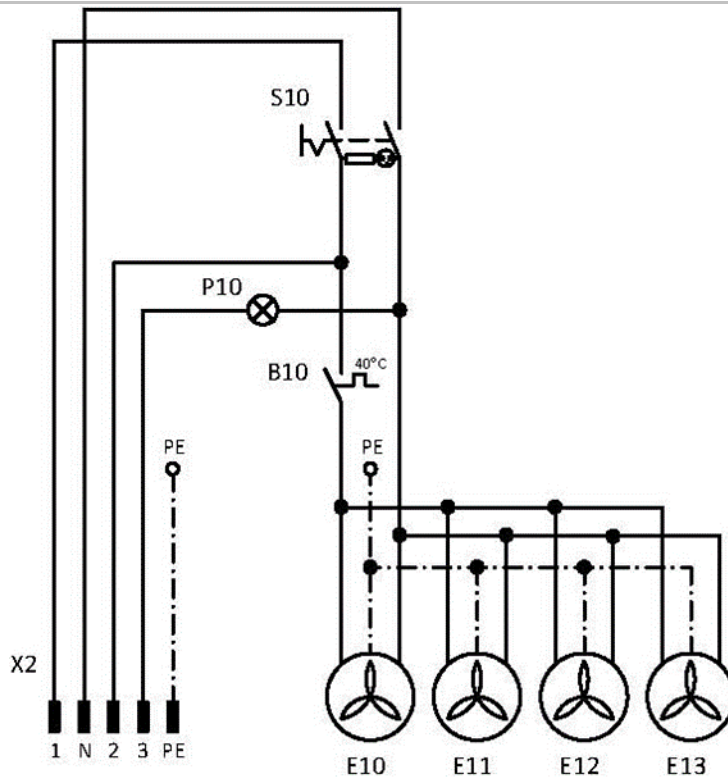1/N/PE~230V, 50/60Hz
 ELEKTRIC OBJECT OF 1st. CAT.

DK50 4VR/50

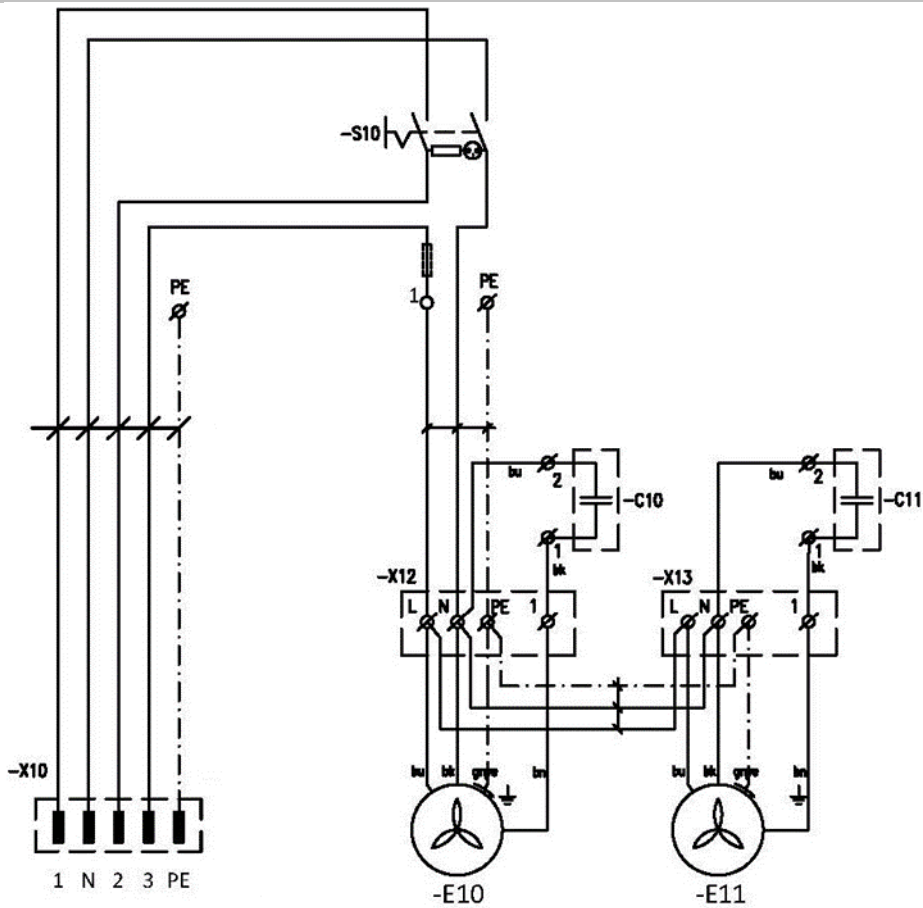DK50 2x4VR/110

Description to electrical diagrams:

M1, M2	Compressor motor	X1	Terminal box
E1 – E4	Compressor fan	X2	Connector
F3	Fuse	M11, M12	Dyer solenoid valve
F1, F2	Breaker	E5 – E8	Dryer fan
K3	PCB	S10	Switch
B1	Pressure switch	E10-13	Cabinet fan
B2	Temperature switch	X10, X11	Connector
Q11,12	Contactora	P10	LED-SERVIS
P1	Hour counter	B10	Cabinet temperature switch

OPERATION

ONLY TRAINED PERSONNEL MAY OPERATE THE EQUIPMENT!

Risk of electric shock.

In case of emergency, disconnect the compressor from the mains (pull out the mains plug).

Burn or fire hazard.

Portions of the air pump may be hot and reach hazardous temperatures during compressor operation that may harm materials or operating staff.

Warning – compressor is controlled automatically.

Automatic start-up. When pressure in the pressure tank drops to switch-on pressure, the compressor automatically switches on. The compressor automatically switches off, when pressure in the air tank reaches the switch-off pressure.

Potential for damage to pneumatic components.

The working pressure settings for the pressure switch set by the manufacturer cannot be changed. Compressor operation at a working pressure below the switching pressure indicates high air usage (see the Troubleshooting chapter).

Required drying performance can only be achieved when following the defined operating conditions.

Drying performance will decline and the achieved dew point will drop if the dryer is operated at any pressure below the minimum working pressure.

During prolonged operation of the compressor, the ambient temperature around the compressor may increase to over 40 °C. At this point the cooling fan automatically switches on. The fan switches off once the space is cooled to around 32 °C.

16. SWITCHING ON THE COMPRESSOR


Start the compressor (without a cabinet) at the pressure switch (1) by turning the switch (2) to position "I." This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

Start the compressor (with a cabinet) at the switch (4) on the front side of the cabinet, the indicator will turn green. This starts the compressor and fills the tank to the switching off pressure, which then shuts off the compressor.

The compressor operates in automatic mode, switched on and off by the pressure switch, depending on compressed air consumption.

Check the values of switch-on and switch-off pressure using the pressure gauge (3). Tolerance of $\pm 10\%$ is acceptable. Pressure in the air tank must not exceed the allowed operating pressure.

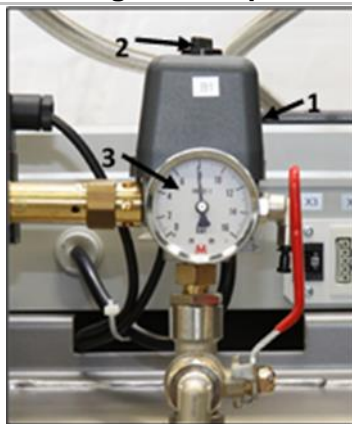Fig. 14: Switching the compressor

Risk of damage to pneumatic parts.

The pressure switch (1) was set up by the manufacturer and any other changes to its settings may be performed only by a qualified technician trained by the manufacturer.

17. SWITCHING OFF THE COMPRESSOR

- Switching off the compressor due to service or any other reason shall be performed using the pressure switch (1) by turning the switch (2) to position „0“ (Fig. 14) and pulling out the mains plug from the socket. This disconnects the compressor from the mains supply.
- Vent the pressure in the air tank to zero by opening the drain valve.

PRODUCT MAINTENANCE

18. PRODUCT MAINTENANCE

The operator should carry out device checks regularly in the intervals defined by applicable regulations. Test results must be recorded.

The equipment has been designed and manufactured to keep maintenance to a minimum. The following work must be performed to preserve the proper and reliable operation of the compressor.

Unauthorised interference hazard.

Repair work outside the framework of standard maintenance (see Chapter 18.1) may only be performed by a qualified technician (an organisation authorized by the manufacturer) or the manufacturer's customer service.

Standard maintenance work (see Chapter 18.1) may only be performed by the operator's trained personnel.

Only use manufacturer-approved replacement parts and accessories.

Danger of injury or equipment damage.

Prior to commencing compressor maintenance, it is necessary to:

- **check if it is possible to disconnect the compressor from the appliance in order to avoid any risk of injury to the person using the appliance or other material damage;**
- **turn off the compressor;**
- **disconnect it from the mains (pulling the cord out of the mains socket);**
- **vent the compressed air from the air tank.**

Venting compressed air poses an injury hazard.

Wear eye protection, i.e. goggles, when venting compressed air from the compressed air circuit (air tank) and from the dryer chamber.

Burn hazard.

When the compressor is running or shortly thereafter, certain portions of the air pump, the compressor's compressed air system, parts of the dryer and connecting hoses between the compressor and dryer may be hot - do not touch these components.

The removed grounding conductor during service must be connected back to the original position after completing the service.

- For any maintenance or repair work, the compressor may be removed from the cabinet by pulling it from the cabinet (on rolling casters as much as the cord between the compressor and the cabinet allows). (see chap. 18) The required maintenance or repair work may then be performed.
- If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug, see chap. 13.2), the compressor shall not be functional. In order to bring the compressor into function again, please follow, see chap. 13.2 or 18.10 or 18.11.

18.1. Maintenance intervals

Time interval	50 Hz	Once a day	Once a week	Once a year	Once every 2 years	2000 h	4000 h	6000 h	8000 h	10000 h	12000 h	20000 h	Chap. replacement	Set of replacement parts	Perform ed by	
															operator	qualified technician
Check of product operation		x											18.2	-		
Drain condensate from air tank – at high humidity		x											18.5	-		
Drain condensate from the air tank - at normal humidity			x										18.5	-		
Check of product function			x										9	-		
Check of pneumatic connections leakage and device inspection				x									18.3	-		
Inspection of electrical connections				x									18.4	-		
Check of cooler and fan				x									18.9	-		
Check of safety valve				x									18.6	-		
Replacement of pump inlet filter and pre-filter 4VR ^{a)}					x									025200139-000		
								x					18.7	025200150-000		

a) data is in hours, if not available, data is in years

b) check the operation of the relief valve on the dryer after every dryer malfunction

c) whichever occurs first: 4 years/8000 hours

d) whichever occurs first: 8 years / 20000 hours

Time interval	50 Hz	Once a day	Once a week	Once a year	Once every 2 years	2000 h	4000 h	6000 h	8000 h	10000 h	12000 h	16000 h	20000 h	24000 h	Chap. replacement	Set of replacement parts	Perform ed by
Conduct a "repeated test" per EN 62353					x										18	-	qualified technician
Filter element replacement in the dryer filter				x											18.8	025200305-000	qualified technician

- a) data is in hours, if not available, data is in years
- b) check the operation of the relief valve on the dryer after every dryer malfunction
- c) whichever occurs first: 4 years / 8000 hours
- d) whichever occurs first: 8 years / 20000 hours

18.2. Check of product operation

- Check air pump condition – the air pumps should be operating normally without excessive vibration or noise. Troubleshoot any problem or call in service personnel if trouble is detected.
- Visually inspect fan operation – the fans must be operating when the air pumps are running. Troubleshoot any problem or call in service personnel if trouble is detected.
- Check to ensure the power cord, the connecting compressed air hoses are undamaged. Replace damaged components or call in service personnel.
- Check the ambient temperature – the ambient temperature must be below the temperature limit (40°C). Cool the space if the temperature is high.
- Compressor with air dryer - open the plug on the condensate collecting vessel and drain the condensate.
- Check the operating condition of the equipment (see Chapter 18.4).

18.3. Check the compressed air connections for leaks and inspect the equipment

Leak testing

- Check the compressor's compressed air lines for leaks during operation – pressure supplied by the compressor.
- Use a leak analyser or soapy water to check all joints and connections for leaks. Tighten or reseal the connection where leaks are found.

Inspecting the equipment

- Check the condition of the compressor aggregate for normal operation and noise levels.
- Fan operation check – the fans must be running during the defined compressor work cycles.
- Check function of the temperature switch (B2) – heat the temperature switch to temperature above 40°C (e.g. using a heat gun – Be careful, do not operate with high temperature on plastic components in the surrounding, it can cause deformation of plastic materials). Fan E1, E3 - the compressor must be under voltage.
- Check filters condition – filters must be undamaged and sufficiently clean.
- Check condition of the pump itself, check if there is no dirt inside the crankcase or clearance in the crankshaft.
- Replace any defective parts if there are any failures.
- Replace any defective parts as needed.
- Check the operating condition of the equipment (see Chapter 18.4).

18.4. Inspection of electrical connections

Risk of electric shock.

Inspect the product's electrical connections when the mains are disconnected.

- Check mechanical function of the main switch.
- Check if the power cable and conductors are not damaged.
- Visually check if cables are connected to the terminal box.
- Check all screw connections of the green-yellow PE grounding conductor.

18.5. Condensate drain

Compressors

Draining the condensate from the pressure vessel is recommended during regular operation. Switch off the compressor at the mains. Reduce air pressure in the device to max. 1 bar by releasing air via a connected device. Install the hose from the basic package into the drain valve. Put the vessel under the hose and release the condensate from the tank by opening the release valve (1).

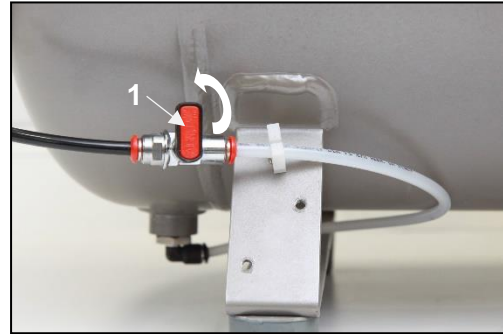
Fig. 15: Condensate drain

Open the drain valve carefully and slowly. If the drain valve is opened too much, it is accompanied by high noise level and uncontrolled jet of the accumulated condensate.

A wet floor resulting from overflow from the vessel poses a slip hazard.

Compressors without a dryer may be equipped with the Automatic Condensate Drain to allow the condensate to drain without operator intervention.

Compressors with air dryer

Condensate from compressors with air dryers is automatically drained into a vessel to collect condensate.

Fig. 16: Check of the condensate collection vessel

Monitor the level in the vessel using the 1 l or 2 l markings (depending on the volume of the vessel), and empty at least once a day. The vessel may overflow if the condensate is not drained out of the vessel at the defined interval.

Before the following checks it is required:

- For compressor variant with cabinet – unlock the door lock and open the cabinet door.

18.6. Check of safety valve

- Turn the screw (2) on the safety valve (1) several times to the left until the safety valve releases the air.
- Let the safety valve vent for a few seconds.
- Turn the screw (2) fully to the right, the valve must be closed now.

Fig. 17: Check of safety valve

Damage to the safety valve could cause pressure to rise to hazardous levels.

Never use the safety valve to release the air pressure in the air tank. This could damage the safety valve. The valve is set to the maximum permitted pressure by the manufacturer.

Never adjust a safety valve.

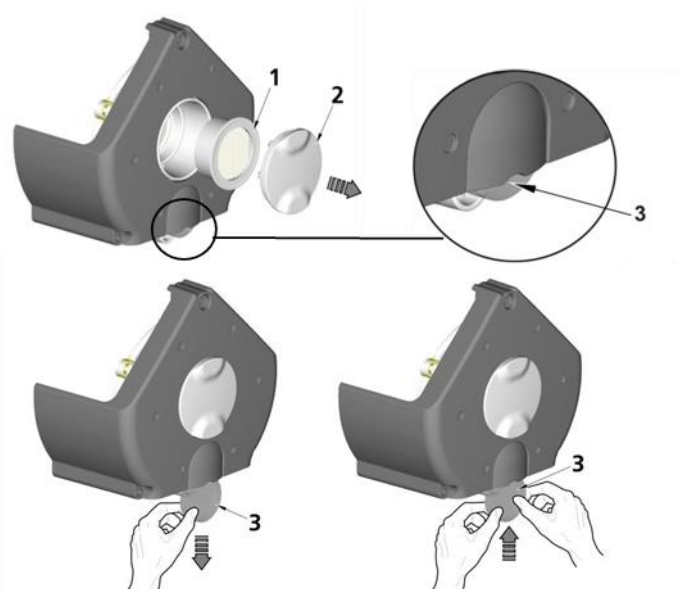
Venting compressed air poses an injury hazard.

Wear safety glasses when inspecting a safety valve.

18.7. Inlet filter replacement

Inlet filter replacement:

- Pull out the rubber plug by hand (2).
- Remove the dirty intake filter (1).
- Insert a new filter and replace the rubber plug.

Pre-filter replacement:

- Pull out the pre-filter by hand (3).
- Replace with a new one and insert it back.

Fig. 18: Inlet filter replacement

18.8. Filter element replacement

- Remove the hose (1) from the quick connector.
- Use a wrench (2) to release the filter vessel (3) and remove.

- Pull down on the filter element (4) to remove.
- Insert a new filter element.
- Re-install the filter vessel.
- Gently tighten the filter vessel with the wrench.
- Re-install the hose on the quick connector.

Fig. 19: Filter element replacement

18.9. Cooler and fan check

The equipment, in particular the compressor fan, cooler fan, and the cooler, must be kept clean to ensure efficient drying. Remove dust from the surface of the cooling fins and fans by vacuuming or blowing down with compressed air.

18.10. Procedure for connecting a compressor disconnected from the cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor (without cabinet) requires that the jumper is always mounted in the terminal strip for correct operation (Fig. 20 pos. C,D, Fig. 21 pos. D). It then replaces the switch on the cabinet.

If there is no jumper in the terminal strip of the compressor (disconnected from the cabinet), the compressor will not work!

If the cord between the compressor and the cabinet becomes disconnected (by pulling the mains plug) and the compressor is removed from the cabinet, the compressor will not be functional. This is why it is necessary to first create a connection to the terminal strip with a jumper (replacing the circuit breaker function) (Fig. 20 pos. A,B, Fig. 21 pos. A).


PROCESS:

Jumper installation (A-B-C-D process)

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is not in the terminal strip – A.
- Lift up the fuse cover – B.
- Insert the jumper into the terminal strip – C.
- Close the fuse cover – D.
- Reinstall the cover on the electrical panel.


- Connect the device to the electrical mains.
- Activate the compressor by turning on the switch on the pressure switch.

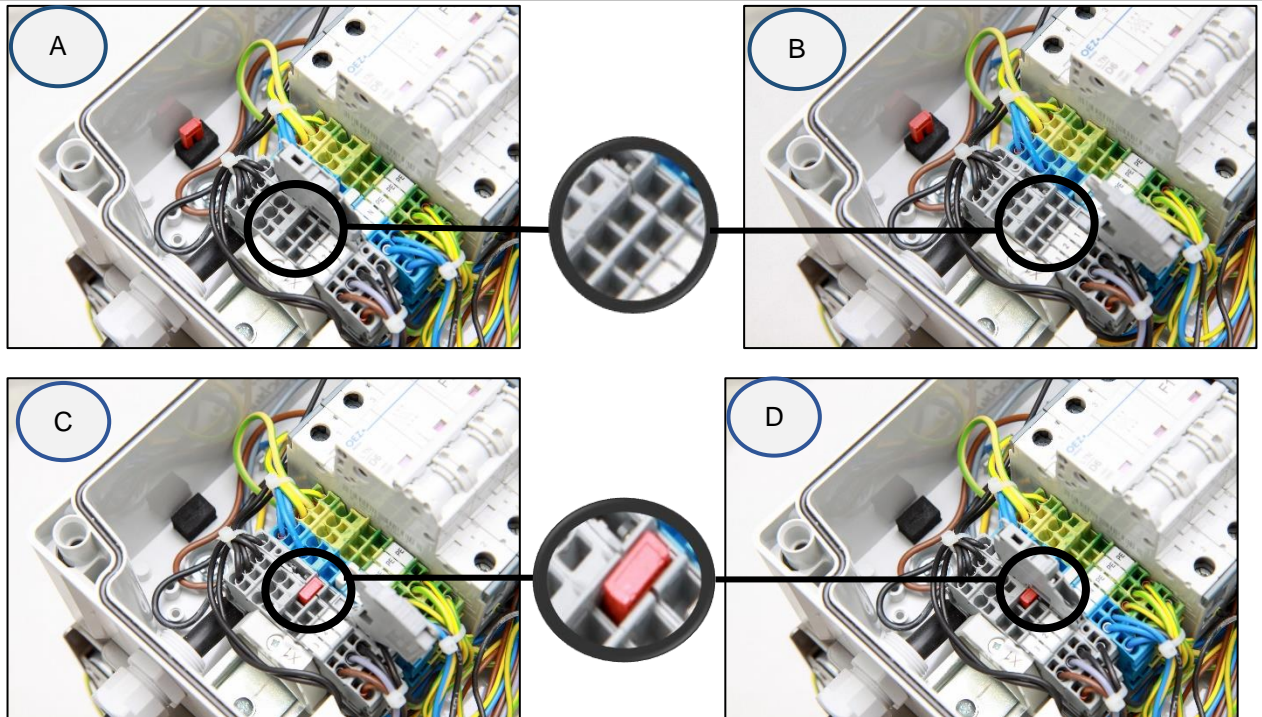
Fig. 20: DK50 4VR/50

400V

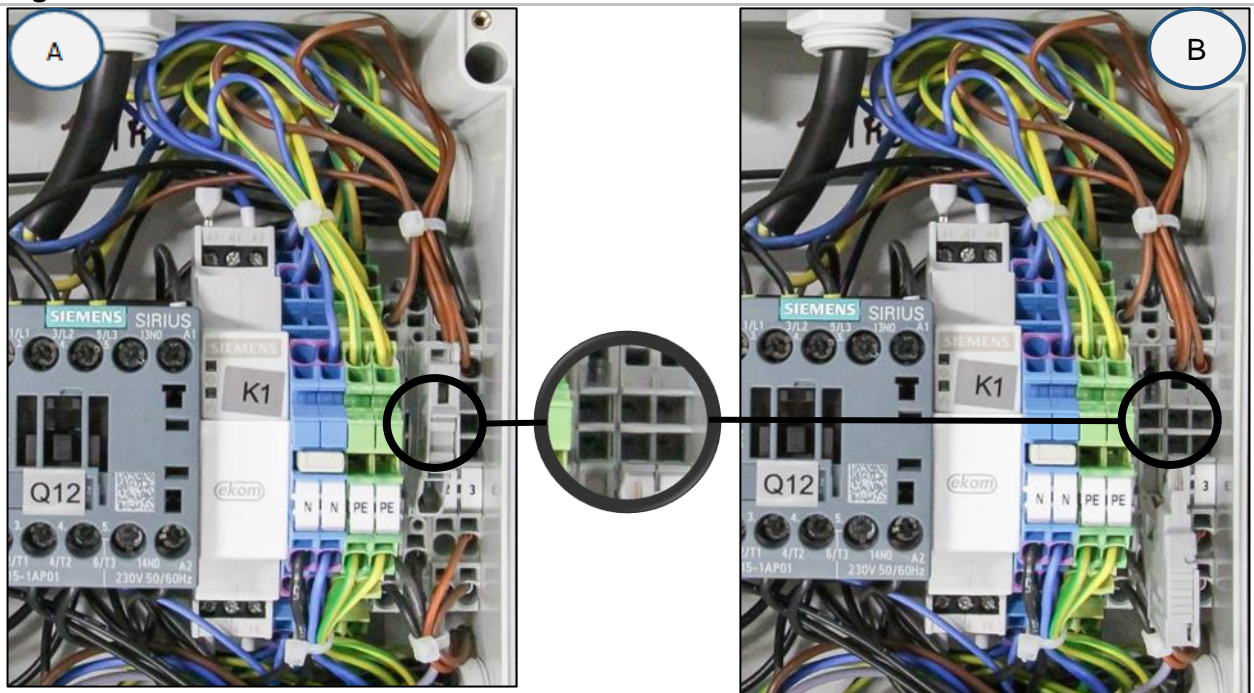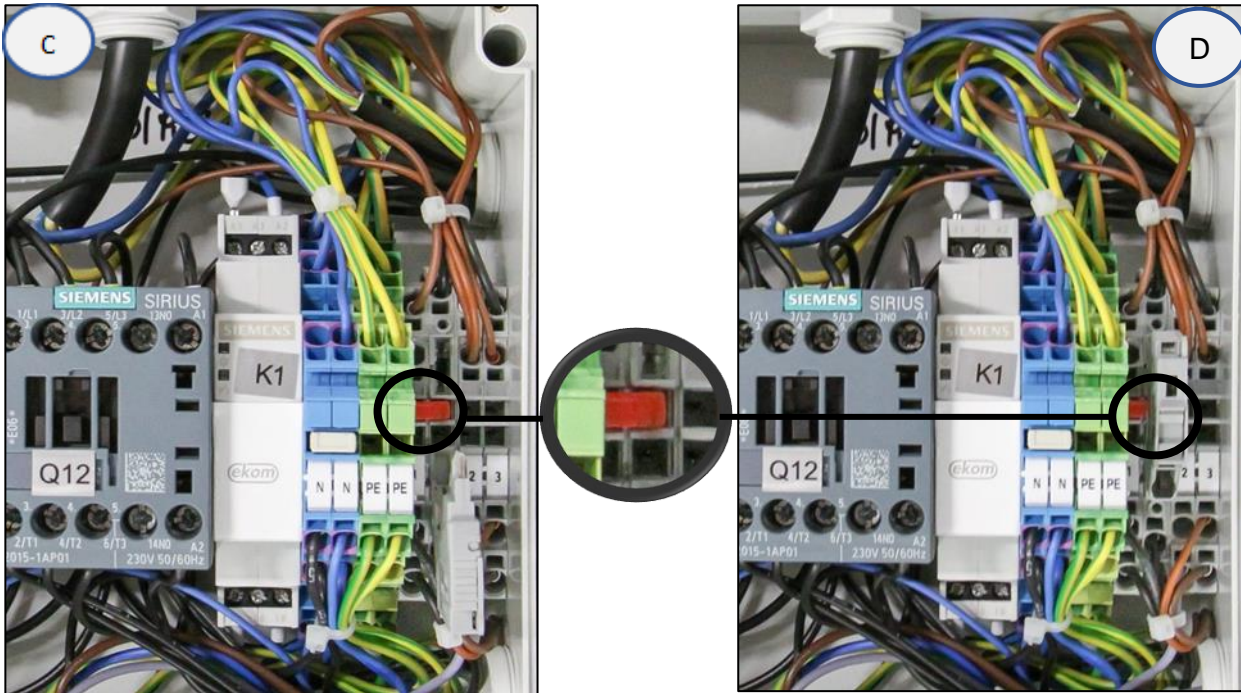Fig. 21: DK50 2x4VR/110

400 V

18.11. Procedure for connecting a compressor to a new cabinet

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

The compressor in a cabinet requires that the jumper is not mounted in the terminal strip for correct operation (Fig. 20 pos. A,B, Fig. 21 pos. A). The switch on the cabinet switches the entire product on and off, including the compressor.

If there is a jumper in the terminal strip of the compressor connected to a cabinet, the switch on the compressor cabinet will not work!

If we want to connect a new cabinet to a compressor that previously worked alone (a compressor without a cabinet), then it is necessary to cancel the connection in the compressor terminal strip by performing the procedure, see further. (Fig. 20, Fig. 21). (See also chap. 13.1 and 13.2).

PROCESS:

Jumper removal (D-C-B-A process)

- Disconnect the product from the mains by pulling the plug out of the socket.
- Remove the cover on the electrical panel (on the compressor).
- The jumper is in the terminal strip – D
- Lift up the fuse cover – C
- Remove the jumper from the terminal strip – B
- Close the fuse cover – A
- Reinstall the cover on the electrical panel.

- Insert the compressor into the box.
- Connect the cabinet to the compressor by inserting the power cord with connector into a power socket. (Fig. 13)
- Connect the device to the electrical mains
- Activate the compressor by turning on the switch on the pressure switch and the switch (4) on the cabinet (Fig. 14).

Compressor with cabinet - when operating in the cabinet, there **must not be** an electrical connection. (Fig. 20 pos. A,B, Fig. 21 pos A).

Note: **In case of using a compressor without a cabinet or in case of maintenance, a jumper which is not installed into the terminal strip should be kept** in a designated place next to the electrical panel.

18.12. Cleaning and disinfection of the exterior surfaces of the product

Clean and disinfect the exterior surfaces with neutral cleaning products.

Use of aggressive detergents and disinfectants containing alcohol and chlorides can lead to surface damage and discolouration.

19. LONG-TERM SHUTDOWN

If the compressor is not going to be used for long period, it is recommended to drain all condensate from the air tank and switch on the compressor for about 10 minutes, keeping the drain valve open. Thereafter switch off the compressor using the switch (2) on the pressure switch (1) (Fig. 14), close the drain valve and disconnect the device from the mains.

20. DISPOSAL OF DEVICE

- Disconnect the equipment from the mains.
- Release the air pressure in the pressure tank by opening the drain valve.
- Dispose of the equipment following all applicable regulations.
- Entrust a specialised company to sort and dispose of waste.
- Worn out components have no negative environmental impact.

TROUBLESHOOTING**Risk of electric shock.****Before interfering with the equipment, first disconnect it from the mains (remove the power socket).****Working with pressurised pneumatic components poses a risk of injury.****Before interfering with the equipment, vent the air tank and the compressed air system to zero pressure.****Troubleshooting may only be performed by a qualified service technician.****Damage to the safety valve could cause pressure to rise to hazardous levels.****Never adjust a safety valve.**

Malfunction	Possible cause	Solution
Compressor does not start	No voltage in pressure switch	Check voltage in socket
		Check fuse – replace faulty one
		Loosen terminal – tighten it
		Check power cord – replace faulty one
	Disconnected winding of motor, damaged thermal protection	Replace motor or re-wind it
Compressor often switches on	Faulty capacitor	Replace capacitor
	Seizure of piston or another rotary part	Replace damaged parts
	Pressure switch does not switch on	Check the function of pressure switch
	Air leak in pneumatic distribution system	Check pneumatic distribution system – seal loose joint
Cabinet switch does not switch off the compressor	Leaking non-return valve	Clean valve, replace seals, replace valve
	Greater volume of condensed liquid in pressure vessel	Drain condensed liquid
	Air leak in pneumatic distribution system	Check pneumatic distribution system – seal loose joint
	Worn piston ring	Replace worn piston ring
Prolonged running of compressor	Contaminated inlet filter	Replace contaminated filters with the new ones
	Defective solenoid valve	Repair or change the valve
	Damaged bearing of piston, piston rod, motor bearing	Replace damaged bearing
Compressor is noisy (knocking, metal noises)	Loose or cracked spring	Replace damaged spring

Dryer doesn't dry (condensed water in the air) *	Cooling fan not working	Replace the fan
	Damaged dryer	Check the power source
	Non-functioning automatic condensate drain	Replace the dryer Clean / replace

) * Clean inside surfaces of the air tank thoroughly and remove all condensed liquid after dryer failure.

Check the moisture content of the air exiting the air tank (see the Technical data chapter) to prevent damage to connected downstream equipment.

21. REPAIR SERVICE

Warranty and post-warranty repairs must be done by the manufacturer, its authorized representative, or service personnel approved by the supplier.

Attention.

The manufacturer reserves the right to make changes to the equipment without notice. Any changes made will not affect the functional properties of the equipment.

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1. Product: (model) DK50 4VR/50 DK50 4VR/50S DK50 4VR/50/M DK50 4VR/50S/M DK50 2x4VR/110 DK50 2x4VR/110S DK50 2x4VR/110/M DK50 2x4VR/110S/M		2. Serial number:	
3.1. User's name:			
3.2. Address of installation:			
4. Equipment connected to the compressor:			
5. Installation / Commissioning:		6. Contents of operator training:	
Product completeness check **	Y	Description of the product and functions**	Y
	N		N
Documentation completeness check **	Y	Product operation: turning on/off, controls, control procedures, data on the display panel, alarms, operation in alarm conditions**	Y
	N		N
Installation/connection to equipment **	Y	Product maintenance: maintenance intervals, maintenance procedure, service intervals, operating activities**	Y
	N		N
Functional test **	Y	Safety measures, warnings – their meaning and compliance **	Y
	N		N
Notes:			
7. Operator instructed on safety measures, operations and maintenance:			
Name:		Signature:	
Name:		Signature:	
Name:		Signature:	
8. Installation and instruction performed by: First name/Last name		Signature:	
Company:		Address:	
Phone:			
Email:		Date:	
9. Distributor:			
Company:		Address:	
Contact person:			
Phone:		Email:	

** mark with an "X" in points 5 and 6 (Y - yes /N - no). Enter any observations from points 5 and 6 into the "Notes" section

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